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## lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2006 Medical Depot, Inc. All rights reserved.  
Drive is a trademark of Medical Depot, Inc.  
Port Washington N.Y. 11050 USA Made in China

walker tray

bandeja para andador

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drive  
MEDICAL DESIGN & MANUFACTURING

[www.drivemedical.com](http://www.drivemedical.com)

## assembly instructions

### PLEASE READ COMPLETELY BEFORE ASSEMBLY AND USE

1. While facing front of walker, grasp sides of tray and clamp brackets on walker front frame.
2. Make sure that the lower clamps are positioned and rest on top of the lower cross brace of the front frame (Figure 1).
3. Check all clamps to assure secure attachment (Figure 1).
4. With thumb and tips of fingers, pull the tray out and up until locked for use position.
5. To stow tray, lift up and push forward and tray will drop down.

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© 2006 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.