

lifetime limited warranty

Your Drive branded product is warrantied to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2014 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington, N.Y. 11050 USA Made in China

free-standing bariatric trapeze with base

item # 13049SV




www.drivemedical.com

assembly instructions

1. Connect sections 1 & 3 by sliding section 3 over section 1 and depressing push pins to secure in place. Repeat for section 2.
2. Connect sections 3 & 4 by sliding section 4 down over section 3. Pull locking pin (A) outward to allow section 4 to lock into position.
3. Slide ring (B) over the tube of section 4 by depressing push button on section 4 and sliding into place. Insert thumb screw (C) through ring (B) and turn clockwise until tight to secure in place. NOTE: Thumb screw should be inserted into the side of B without the receptacle nut to be sure it locks into place correctly.

The Trapeze , used in conjunction with a bed, can assist an individual weighing up to 650lbs (295kg). The Trapeze is designed to provide support, increase stability and assist the user when repositioning himself/herself in bed. The Trapeze is not designed, however, to support the total body weight of an individual. Use it for assistance only.

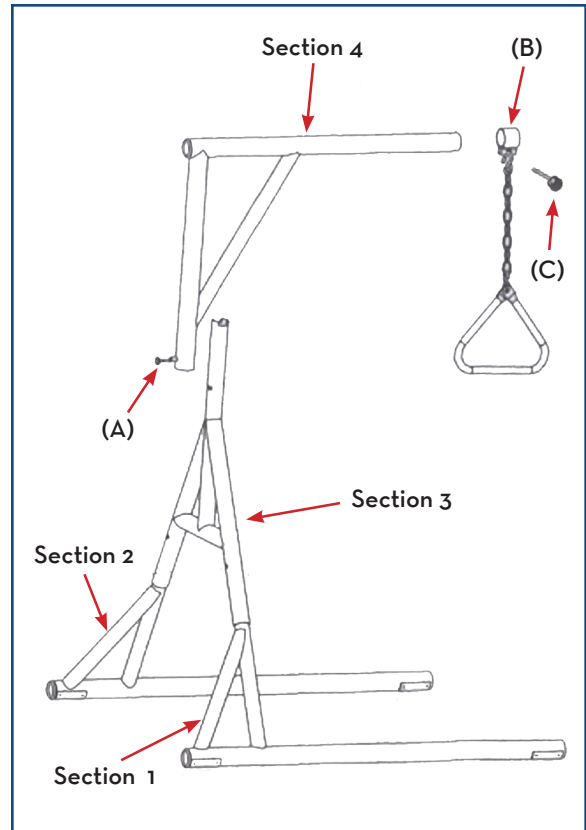

IMPORTANT

Prior to patient use, be sure that all connections are inspected and secure.