



There is Power Mobility. And then there's  
**POWERFUL  
MOBILITY**



move-x

Call 1-800-944-1997 or visit us at [www.evrider.com](http://www.evrider.com)



## The work of Art – *Move X* Rollator

### The most compact Rollator in the world

The patented German design of latest generation of rollator is amazingly light, comfortable and durable. X folding in a second on your index fingers makes the rollator extremely compact for transportation. This work of art is like a luxury car, which makes you feel confident using it. With Move X, you will no longer find it difficult to go out. It's a beautiful day, go out and enjoy. Move to the wonderful life.





### **Status:**

1. In use
2. Storage
3. carrying

1



2



3



### **Features:**

- Very easy and fast to fold and unfold
- Extremely light weight
- Excellent brakes, curb lifter and edge guard
- Stands securely folded for easy storage
- Extremely compact when folded for transportation.
- Excellent for users who travel by car, train, bus or airplane.
- 12 ball bearings in the wheels and forks provide excellent driving comfort.
- Can be cleaned.
- All wear and tear parts can be replaced and brakes can be adjusted.
- Tested and approved according to EN ISO 11199, certified by TUV








## ***Folding procedure***

1

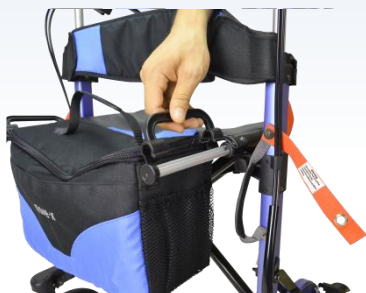

**Take off the shopping bag (better but not a must)**

To grab the bag handle and take off the bag

2



**Push in the bag holding pin (better but not a must)**

When bag is taken off, push the holding pin inside

3



**Pull up the folding webbing**

The folding webbing is knitted at the left hand side of the seat.  
Pull it to fold the rollator horizontally to the **Storage status**



## ***Folding procedure***

4



### **Pull the X folding handle**

Use your fingers to pull up the X-folding handle at each side of the rollator handle

5



### **Buckle for fixation**

After folding, use the buckle webbing to fix the rollator

***Reverse procedure for unfolding***



## Specification

Weight Capacity: 125kg  
Product Weight: 7.4KG  
Bag Capacity: 5kg



Rear wheels distance:  
600mm



Front and rear wheel  
Distance: 460mm



Front wheels distance:  
535mm(20")



Handle height:  
775-895mm



Seat height:  
580mm



Seat width:  
380mm(15")



Handles distance:  
470mm



Front and rear  
wheel size:8"



Weight capacity  
135kg



Folded size  
640mm(Closed length)  
280mm(Closed width)  
390(Closed height)





## *Spotlight on details*

