There is Power Mobility. And then there's

POWERFUL MOBILITY





Call 1-800-944-1997 or visit us at www.evrider.com

FOLDING PROCEDURE

How to unfold:

- 1. Release the folding webbing button. (Figure 1)
- 2. Hold the handle and pull up until a click sound to ensure it's good to be used. (Figure 2)
- 3. Push the safety lock down to lock the folding handle.
- 4. Push the seat side tubes down until a click sound. (Figure 3)
- 5. Adjust the handle to the proper position.
- 6. Pull out the bag holding rod.
- 7. Clip the bag on the bag holding rod and attach the hook on the seat loop.
- 8. Push down the castor lock to free the castor .







How to fold:

- 1. Pull up the front castor lock to fix the direction of caster(better but not a must).
- 2. Take off the shopping bag (better but not a must): To grab the bag handle and take off the bag.
- 3. **Push in the bag holding rod (better but not a must)**: When bag is taken off, push the holding rod inside. When bag is taken off, push the holding rod inside
- 4. Adjust the handle to the lowest position.
- 5. **Pull up the folding webbing**: The folding webbing is knitted at the left hand side of the seat. Pull it to fold the rollator horizontally to the Storage status. (Figure 1)
- 6. Push the button of folding webbing on the seat tube button to fix the folding position.
- 7. Pull up the safety lock on outer side of the handles to release the safety lock.
- 8. **Pull the X folding handle**: Use your fingers to pull up the X-folding handle at each side of the rollator handle. (Figure 2)
- 9. Fold down the rollator frame. (Figure 3)
- 10. **After folding, use the buckle webbing to fix the rollator:** Put the webbing on the rivet each side to secure the folded position. (Figure 4)









FEATURES


Brake

Pull up the brake lever to activate running brake. Push down the brake lever to activate parking brake.



Handle height adjustment

- 1. Pull up the fixation lock to release.
- 2. Pull the fixation knob and turn 90 degree.
- 3. Adjust the handle to proper position.
- 4. Turn the fixation knob 90 degree and let go the knob to fixation position and a click sound. (If no click sound, fine tune the handle for a click sound and it shows the handle is fixed at proper position)
- 5. Try the handle to ensure it doesn't move
- 6. Push back the fixation lock to tighten the handle.

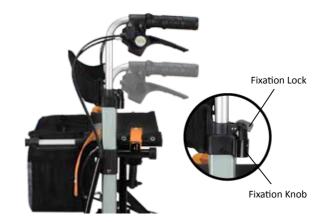

Folding mechanism

Pull to disengage the rollator frame to start folding.

The safety lock

Up position: prepare to fold the rollator Down position: After unfolded, push down the safety lock to secure the folding handle









Cane & Umbrella Holder



Installation of Basket

- 1. Pull out both tubes attached to the seat and hook the basket up.
- 2. Hear the click sound to ensure the basket is secured onto the tube.
- 3. Attach the hook on the seat loop.





EN Instruction EN Instruction

SPECIFICATION











Osann Move X

Test according to ISO11199-2 Weight Capacity: 250 lbs Bag Capacity: 11 lbs Overall Dimension: L22 X W23.5 X H31~36 inch Folded Dimension: L23 X W11 X H15 inch Seat Dimension: W16.5 X D7.5 X H22.5 inch Backrest:L14.5 X W10 inch



CLEANING METHOD

- 1. Wipe down your equipment weekly (or more often as needed) with a cloth dampened in a mild detergent/ water solution.
- 2. Allow the equipment to dry before using or storing.
- 3. Store your equipment safely away in a clean, dry location when not in use.

WARNINGS

FOR PERSONAL SAFETY

- For your health and comfort, be sure to use good posture and hold the rollator so that you can operate the brakes with ease.
- Please store the rollator in a safe environment.
- Unfold the rollator completely into the locked position before use.
- Possible risk of finger trapping do not put fingers or hands into the hinge of the rollator when folding or opening for use.
- Ensure that the wheels are stable and on the ground before using.
- Ensure the brakes are adjusted properly and tested before using.
- Ensure that the seat is locked in position before sitting.
- Always sit in the center or the seat, with the seat adjusted that the user has their feet on the floor.
- Always have the parking brakes locked in position before sitting.
- Never sit on the rollator when it is folded for storage.
- Never stand on the rollator, or use it as a step.
- Never transport a person on the seat of the rollator.
- Use caution when lifting and carrying the rollator.

WARRANTY

ONE-YEAR LIMITED WARRANTY: One (1) year from the date of purchase, if any part or component of the rollator is found upon examination to be defective in material and/or workmanship,it will be repaired or replaced, at EV Rider, LLC's option, without charge. Labor charges are NOT included. The purchaser is responsible for the delivery to the authorized repair facility. Satisfactory proof of purchase, serial number and the date of purchase are always required.

WARRANTY SERVICE can be performed by EV Rider, LLC or by an authorized EV Rider, LLC service facility. **Do not return faulty parts to EV Rider, LLC without prior consent.** A request for aReturn Authorization is required prior to returning items. All transportation costs and shipping damage incurred while submitting units and/or parts for repair or replacement are the responsibility of the original purchaser.

WARRANTY CLAIMS must be reported to:

EV Rider, LLC 61410 Arc Way, suite A Fort Myers, FL 33966 Service@evrider.com 1-800-944-1997 Ext 203 www.evrider.com

SHIPPING COSTS - Domestic: All warranty items will be shipped via Fedex Home/Ground to any destination within the Continental U.S. free of charge. Requested expedited shipping will be at the expense of the purchaser.


SHIPPING COSTS - International: Parts will be provided under warranty at no charge, but shipping to any offshore and/or international location will be the responsibility of the purchaser.

WARRANTY EXCLUSIONS: This warranty is <u>only given to the original purchaser of the Rollator.</u> This Warranty excludes:

- Consumable items which may need replacing due to normal wear and tear (tires, upholstery and seating, plastic shrouds, etc.)
- Damaged caused by: abuse, misuse, accident, or negligence, improper operation, excessive loading, maintenance, or storage, commercial use or use other than normal.
- Used other than for the purpose of the purchaser's mobility.
- Repairs and/or modifications made to any part of the rollator without specific and prior consent from EV Rider, LLC.
- Labor, service calls, shipping, and other charges incurred for repair of the product not approved by EV Rider, LLC.

Implied warranties, including those of merchantability and fitness for a particular purpose, are limited to one (1) year from the date of the original purchase and to the extent permitted by law. Any and all implied warranties are excluded. This is the exclusive remedy. Liabilities for consequential damages under any and all warranties are excluded. Some states do not allow limitations on how long an implied warranty lasts or do not allow the exclusion of limitation of incidental or consequential damages. So, the above limitation or exclusion may not apply to you.

EN Instruction EN Instruction

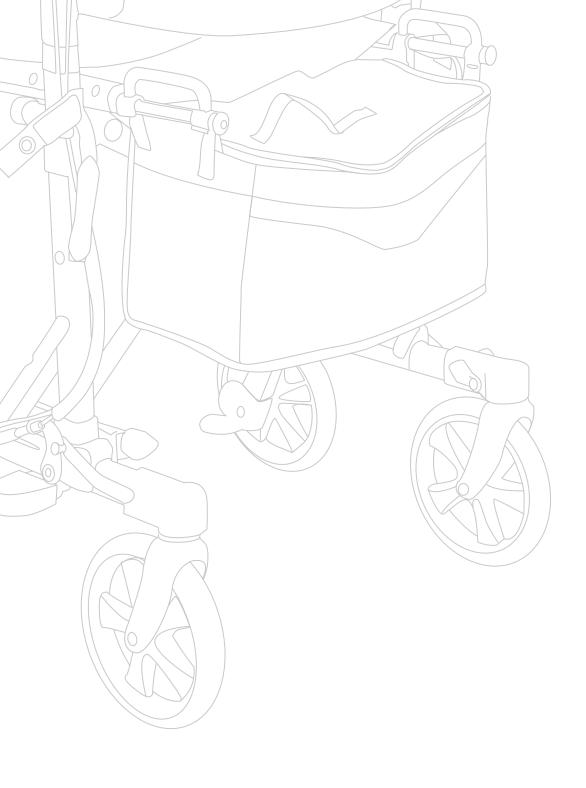

Importer

EV RIDER, LLC 6410 ARC WAY, SUITE A FORT MYERS, FL 33966 www.evrider.com Toll Free: 1-800-944-1997 Phone: 239-278-5054

Fax: 239-278-1431 E-mail: sales@evrider.com