

# ASSEMBLY INSTRUCTIONS:

Remove item from carton and packaging. Be wary as to not dispose of the carton or packaging supplies until the unit is inspected. Return may be denied if original carton and packaging is missing.

1. Unfold the rollator by carefully pushing down on the frame, and folding bar. Press folding down firmly to lock frame into open position.



2. Carefully cut zip ties and remove packaging from handles which have been mounted to the frame to prevent damage during shipping.



3. Install padded backrest into the backrest receivers.



4. Install handles by inserting into the frame. Adjust height accordingly utilizing quick adjust knobs.



5. Install basket/bag beneath the seat.

