GUARDIAN[®]

Family-owned. Trusted by
hospitals for over 40 years.

BASIC ROLLATOR, FOUR-WHEELED ROLLING WALKER USER INSTRUCTIONS & WARRANTY

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LIFETIME
LIMITED
WARRANTY

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Safety Instructions

To ensure your safety in using the Medline rollator, the following safety information and all instructions must be followed.

GENERAL CAUTIONS:

- ⚠ **DO NOT use this product without first reading and understanding the instructions contained in this manual.** If you are unable to understand the warnings, cautions, or instructions, contact a healthcare professional, dealer, or technical personnel before use. Failure to do this may result in bodily injury or damage to the device.
- ⚠ **DO NOT** self-propel the rollator while seated.
- ⚠ Rollators are intended for individual use only and are **NOT TO BE USED AS A WHEELCHAIR.** Serious injury to the user and/or damage to the rollator's frame or wheels may result from improper use.
- ⚠ **DO NOT** use the rollator to walk backwards, down gradients or to climb stairs, curbs, or to go over obstacles. Serious risk of fall or injury may occur.
- ✓ The backrest is intended to provide back support while seated. The backrest is not intended to support full body weight. Avoid leaning while sitting as this may result in fall.
- ✓ All wheels must be in contact with the floor and/or ground at **ALL TIMES.** This will ensure the rollator is properly balanced.
- ⚠ **DO NOT** exceed the maximum weight capacity (including bag contents) or serious injury could result.
Maximum weight capacity: 250 lbs (113 kg)
 - Items should be placed completely in tote bag and should not protrude. Excess weight may cause the rollator to tip, resulting in bodily injury or rollator damage
 - The bag located under the seat has a weight limitation of 10 lbs (4.5 kg).
 - The bag that hangs off the front has a weight limitation of 5 lbs (2.25 kg).
- ✓ Before each use make sure the rollator frame is fully opened.
- ✓ Periodically wipe down the rollator with a soft cloth and nonabrasive cleaner.
- ⚠ **Do not** use rollator like a kneewalker or scooter. This device should only be used for assistance to walk, not hop or any other type of mobility.

SEAT CAUTIONS:

- ✓ The brakes **MUST** be in the locked position **BEFORE** sitting on the seat.
- ⚠ **DO NOT** sit on the rollator while it is being loaded into a vehicle or while it is being transported in a vehicle.
- ⚠ **DO NOT** use the seat to transport people or objects.
- ⚠ **DO NOT** sit on the seat when the rollator is on an incline.

BRAKE CAUTIONS:

- ⚠ Check the brakes for proper operation. Make sure that all parts are secure and that the moving parts are in good working order.
- ⚠ Wheels should roll easily and not wobble.
- ⚠ When rising from a lower position, **DO NOT** use the rollator to provide assistance to the user. If the rollator is used to pull the user up to a standing position, the rollator may move even if the brakes are in the locked position and this could result in a fall.
- ⚠ **DO NOT** push the rollator while brakes are enabled. Brakes are to be used to keep the wheels from rolling. However, it may not prevent the rollator from moving if it is pushed.

Note: Only use accessories and spare parts authorized by Medline Industries

Rollator Parts List

Note: You may wish to request a set up and operating demonstration with your local Medical Equipment provider or pharmacy.

Rollator Parts List

- 1 Rollator Frame
- 1 Backrest with Push-buttons
- 2 Handlebars
- 1 bag
- 2 Front Wheels
- 2 Rear Wheels
- 2 Star Knobs
- 2 Screws and Curved Washers

Figure 1: Assembled Rollator

Assembled Rollator

- A. Handgrips
- B. Adjustable Handlebars
- C. Star Knob
- D. Padded Backrest
- E. Backrest Push Buttons
- F. Padded Seat
- G. Brake Lock
- H. Brake Cables
- I. Folding Support Bracket
- J. 6" (15.2 cm) Front Wheels
- K. 6" (15.2 cm) Back Wheels
- L. Storage Bag

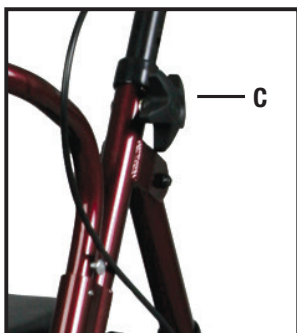
Figure 2: Star Knob

Assembly Instructions

1. Carefully remove the rollator components from the shipping carton. Refer to diagram **(Figure 1)** as needed.
2. Examine all parts for shipping damage. **DO NOT** use the rollator if there is apparent damage or if any parts (other than the bag) are missing. If there are missing or damaged parts, please call Medline Customer Care toll free at 1-800-MEDLINE.
3. Remove plastic cover from wheels. Check all of the wheels to see that they are in good working order and roll freely.
4. Unfold the rollator by pushing down on the Folding Bracket **(I)** until the Frame is in full extension.
5. Insert the padded backrest **(D)** so that the padded portion faces the front of the unit. Push down on the backrest while depressing the the round plastic push button until it snaps into place.
6. Insert the handlebars **(B)** so that the handgrips **(A)** are aligned with the rear wheels. The brake cables should be on the outside of the handlebars and frame. Ensure excess brake cable loops towards the front of the rollator.
7. Adjust the height of the handlebars **(B)** for proper fit. To determine proper height, stand upright behind the rollator with shoulders relaxed, arms hanging at your side and eyes looking straight ahead. Standing in this position, the handgrips **(A)** of the rollator should be adjusted to a height even with the wrists. This will position the arms at an approximate 20° to 30° bend when using the rollator. Secure the handlebars at the closest hole to the desired height. Insert the screw from the inside of the rollator through the frame. Attach the washer so it fits snugly against the frame. Secure the star knob to the screw. **See Figure 2. (Important: Hand Tighten Only!)**
8. Place the bag under the seat.

Operating Instructions

Using the rollator as a walker:

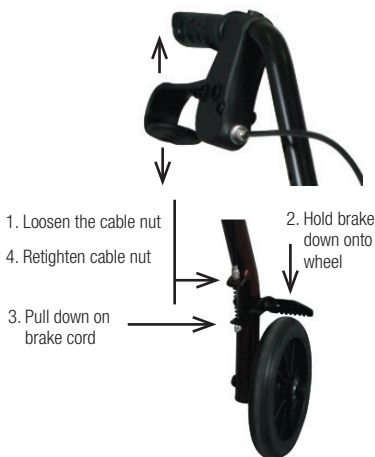
For proper positioning and use of the rollator, imagine a line drawn across the back of the rear wheels. Part of one foot should be slightly ahead of that line as you walk.

Using the rollator as a seat:

- Push the rollator to the desired location.
- **LOCK** the brakes **BEFORE** sitting.

To Operate the Brakes:

- To slow down, squeeze the brake handles up.
- To lock the brakes, push down on the brake handles until you feel or hear the brakes click into the locked position. If the brakes have been adjusted but the rear wheels on the rollator move while the brakes are in the locked position, **DO NOT** use the rollator. Please call our toll-free Customer Service number at 1-800-MEDLINE for returns or repairs.
- To release the brakes from the locked position, simply pull up on the brake handles.

CAUTION: After making any adjustments to the rollator brakes, you must test the brakes before the rollator is used. See Maintenance Instructions.

Maintenance

Your rollator should be checked periodically to ensure the brakes are functioning properly and that all nuts and bolts are secure.

Periodically test the brakes by both walking and squeezing the brakes and by stopping, pushing down on the brakes to a locked position, and then trying to push the walker. If the brakes do not lock the wheels, the brakes need to be adjusted. If adjustment does not tighten the brakes, **DO NOT** use the rollator.

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LIMITED WARRANTY

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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Medline Canada
1-800-396-6996
www.medline.ca | Canada@medline.com

Medline Mexico
01-800-831-0898
www.medlinemexico.com | mexico@medline.com

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2016 Medline industries, Inc. Medline and Guardian are registered trademarks of Medline Industries, Inc.

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