ELECTRIC AND MANUAL BEDS

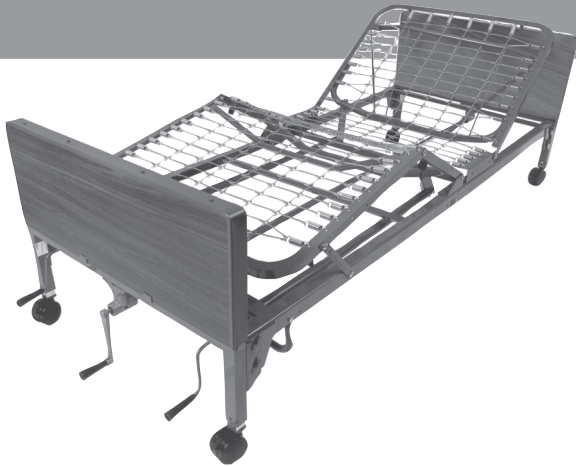
OWNER'S ASSEMBLY AND OPERATING MANUAL

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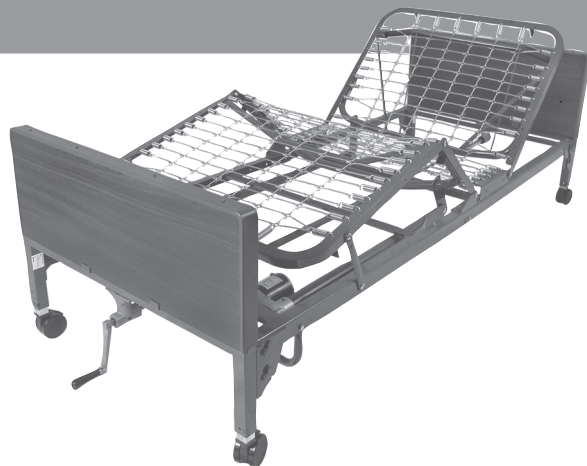
Lit Manuel Manivelle Unique

15004

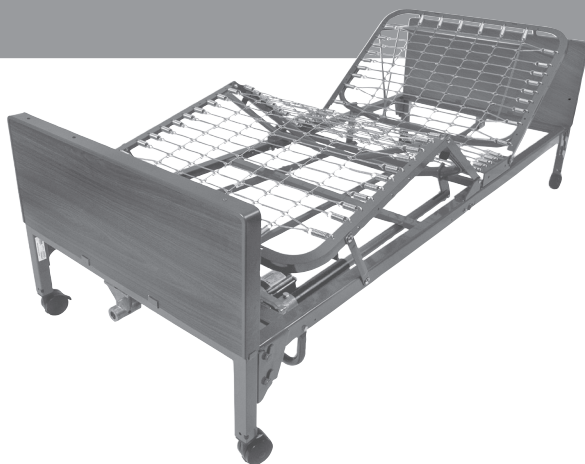
Semi-Electric Bed Single & Two Crank - Hi/Lo

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Manivelas – Alto/Bajo

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Manivelles – Hi/Lo

15005

Full Electric Bed

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drive™

SPECIFICATIONS

PACKAGING/HANDLING

UNPACKING

1. Check bed components for any obvious damage to the carton or its contents. If damage is evident, contact your local Drive dealer.
2. Remove any loose packing from the cartons.
3. Carefully remove all components from the cartons.

The three cartons and contents are as follows:

1. Bed End carton:

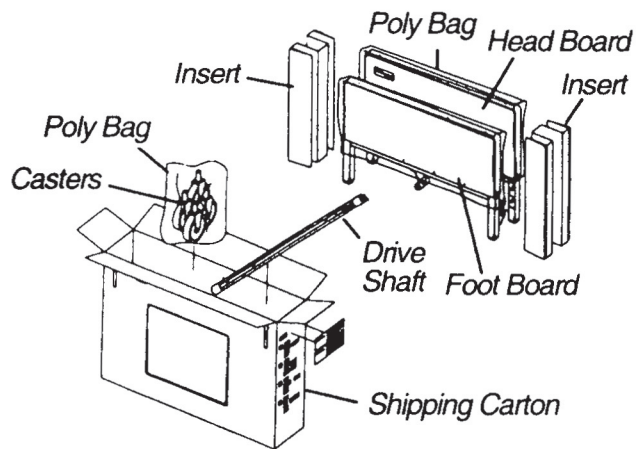
One (1) Head end.

One (1) Foot end (crank assembled on Bed No. 15003)

One (1) Drive Shaft Assembly-two-piece.

- * Four (4) Casters (two [2] locking and two [2] non locking casters).

NOTE: Set the Drive Shaft Assembly aside for Bed Nos. 15003, 15004, 15005

2. Foot Spring carton-Electric models (15012 or 15013):

One (1) Electric foot spring w/junction box and cord.

One (1) Hand pendant control.

One (1) Pendant holder.

One (1) Emergency crank handle.

CAUTION

DO NOT attempt to operate bed controls prior to completion of assembly as damage to bed components or personal injury may occur.

3. Head Spring carton:

One (1) Head spring

INSPECTION

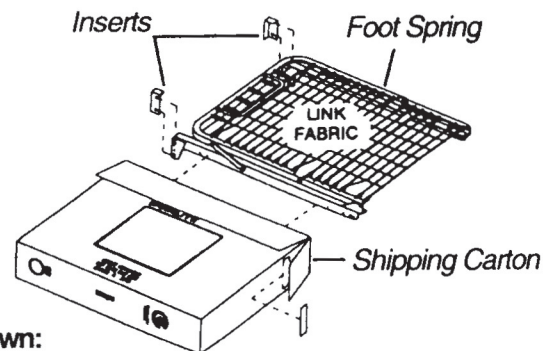
1. Examine each item carefully for scratches, nicks, dents or any other apparent damage.

NOTE: For semi-electric and full-electric beds:


2. Inspect the power supply cord for cuts or damage to the cord and/or plug.
3. Inspect the junction box for any damage to the connectors.
4. Ensure that the motor plugs are in good working order and fit properly into the junction box.

STORAGE

1. If bed is not to be assembled immediately, retain the cartons and packaging for storage until assembly is to be performed.
2. Store the repackaged homecare bed in a dry area.
3. DO NOT place other objects on top of the repackaged cartons.

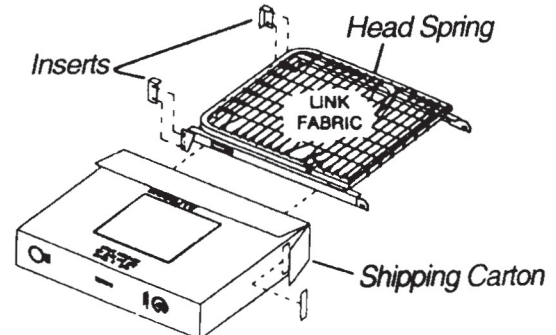Not Shown:
Junction Box w/Cord
Pendant and Emergency Crank Handle

ASSEMBLY

CONNECTING THE SPRING SECTIONS (SEMI/FULL ELECTRIC AND MANUAL BEDS) (FIGURE 1)

1. Remove the head spring (Model 15008) from packaged container and place on its side to your left with the center mounting latches to your right.
 2. Remove the foot spring from packaged container and place on its side to your right with the head spring pull tube at the top of the assembly and the center mounting rivets facing to the left.
 3. For semi-electric (15012) and full-electric (15013) foot spring, the motors should be to your right and for manual foot spring(15009) the crank handles will be to your right.
 4. The head and foot spring should now be on their side at approximately a 90° angle to one another.
 5. Hook the bottom head spring center mounting latches to the bottom foot spring center mounting rivets.
 6. Hook the top head spring center mounting latches to the top foot spring center mounting rivets.
- NOTE: It may be necessary to lift slightly on the head spring or foot spring to secure the mounting latches and rivets.*
7. After head and foot springs are connected, push the springs into a horizontal (straight) position while still on their sides.

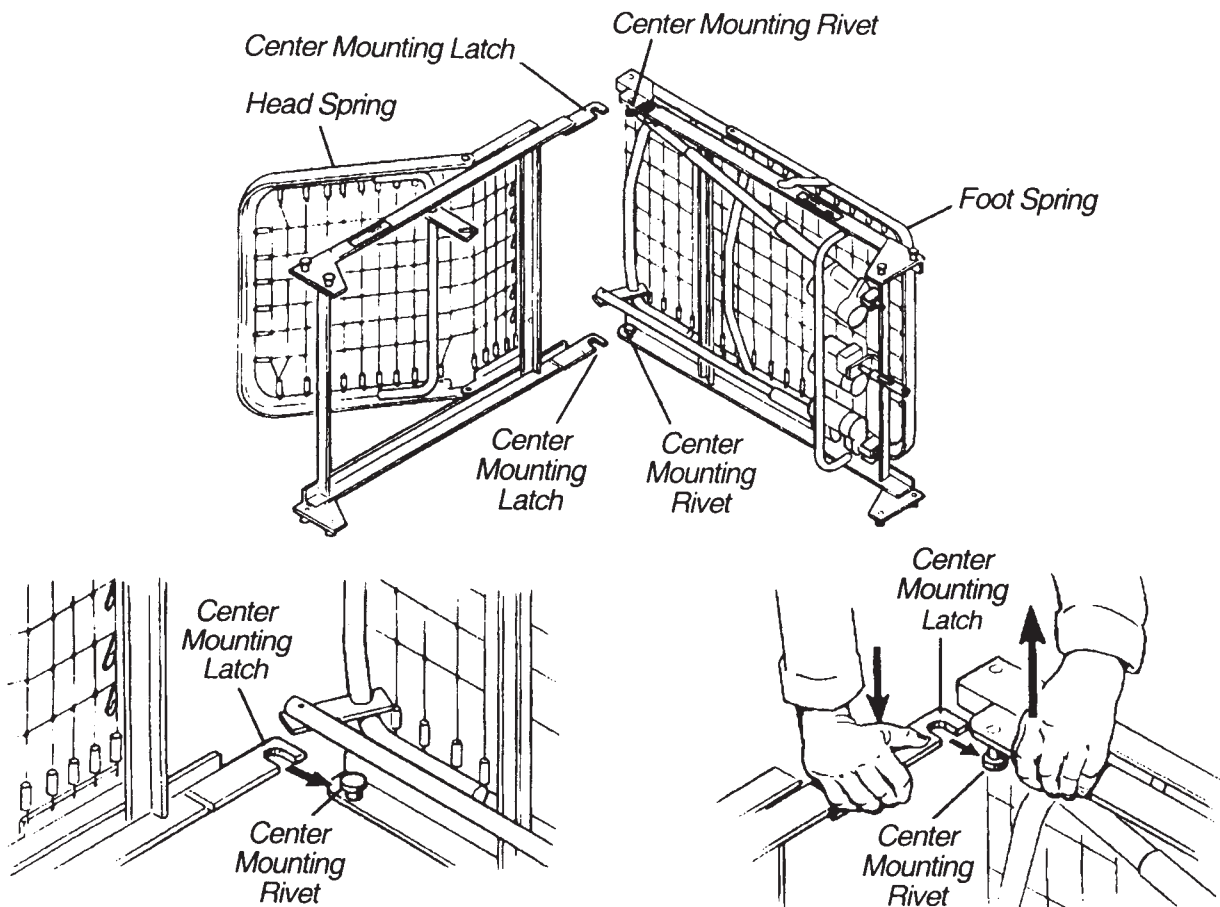
FIGURE 1 - CONNECTING THE SPRING SECTIONS

ASSEMBLY

CONNECTING THE SPRING FABRIC (FIGURE 2)

1. Lift the head spring upward to give slack to the fabric, and connect the spring fabric together with the links provided.

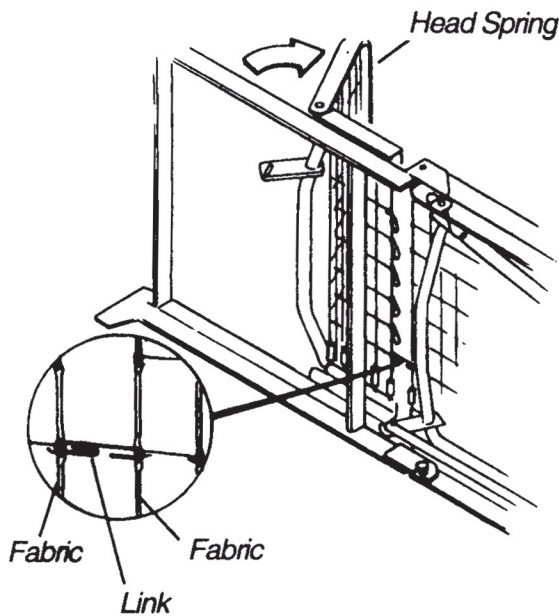
FIGURE 2 - CONNECTING THE SPRING FABRIC

ADJUSTING AND RECONNECTING THE HEAD SPRING PULL TUBE (FIGURE 3)

1. To provide up/down movement of the head section, disconnect the head pull-tube assembly from the side frame of the foot spring by removing the hitch pin, grommet and washer from the clevis pin.
2. Extend the inner pull tube shaft manually until the spring button "clicks" into the adjustment hole of the outer pull tube.
3. The spring button should protrude through the adjustment hole and the clevis pin of the now extended pull tube arm will line up with the lift arm slot.
4. Connect the pull tube end assembly to the lift arm of the head spring by inserting the clevis pin into the lift arm slot; reconnect the grommet and washer. Secure with hitch pin.

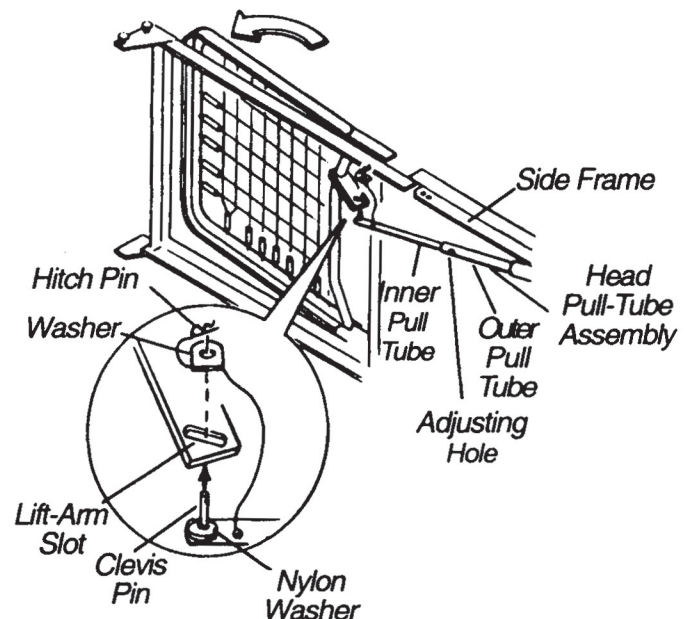
FIGURE 3 - ADJUSTING AND RECONNECTING THE HEAD SPRING PULL TUBE

ASSEMBLY

ASSEMBLING THE BED ENDS (FIGURE 4)

NOTE: When connecting the bed ends (headboard and footboard), remember that the headboard is the taller of the two.

1. Stand the headboard as close to the head spring as possible.
2. With one hand, grasp the head spring frame and raise it until the rivets on the corner plates of the spring are at the correct height to fit into the corner locks on the headboard.
3. Tilt the headboard backward slightly and the rivets should slide into the corner locks more easily.
4. When the headboard is returned to its full upright position, the headboard will lock into place.
5. Repeat the same procedure for the foot end section.

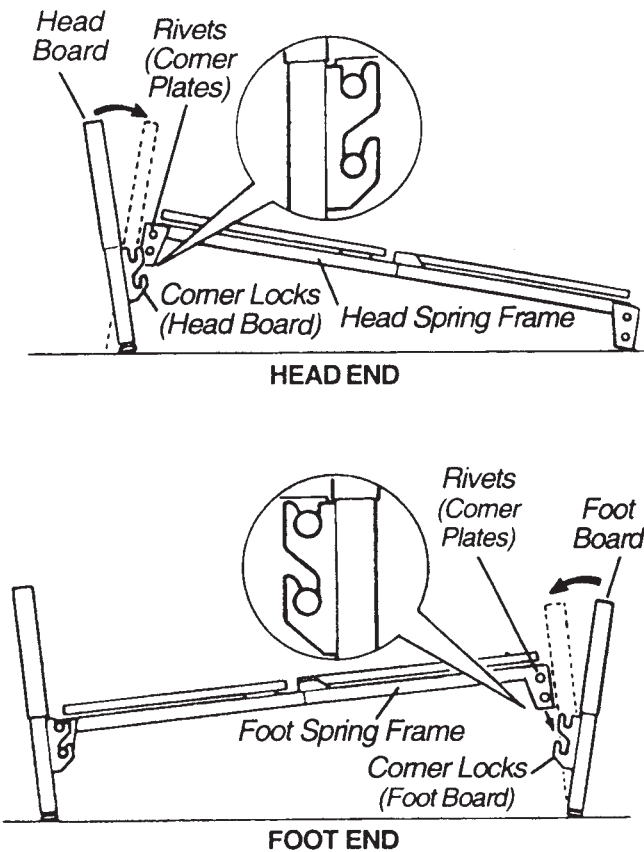
FIGURE 4 - ASSEMBLING THE BED ENDS

ASSEMBLY

INSTALLING THE CASTERS (FIGURE 5)

1. Install casters by carefully lifting the bed ends approximately 5-inches (12.7 cm) from the floor and insert the shaft of caster assembly into the caster socket.

NOTE: Position the two (2) locking casters diagonally opposite from one another.

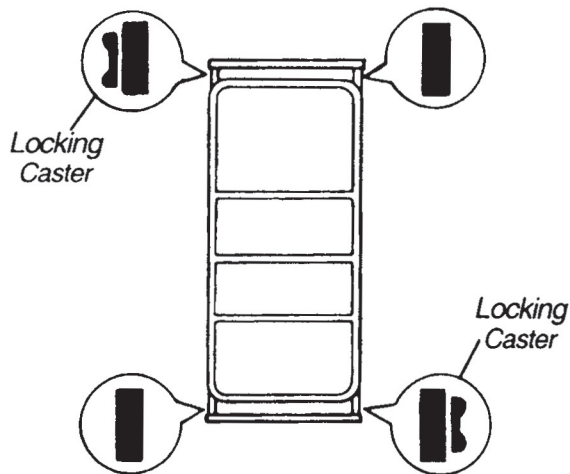
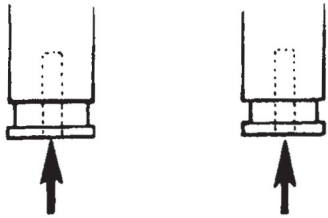
FIGURE 5 - INSTALLING THE CASTERS

DRIVE SHAFT ASSEMBLY AND INSTALLATION (FIGURE 6)

The drive shaft is composed of two (2) sections. The inner shaft which has a positioning spring button, and the outer shaft, which has two (2) positioning holes and one (1) storage position hole. Together they make the telescoping drive shaft used on all manual and electric Hi/Lo model beds.

1. Remove the plastic caps from each end of the drive rod.
2. Depress spring button on the inner shaft and insert the inner shaft into the outer shaft.
3. Slide the inner shaft to the desired position for the type of bed being used and engage the spring button into the proper positioning hole.
4. For semi-electric and manual beds, engage the spring button into the furthestmost positioning hole in outer shaft.
5. For full-electric models, engage the spring button in to the positioning hole in the outer shaft located next to the semi-electric/manual positioning hole.
6. Connect the drive shaft to the bed by attaching either end of the drive shaft to the head end gear box.
7. Connect the other end to the Hi/Lo motor output shaft and then connect the foot end gearbox output shaft to the Hi/Lo motor output shaft spring-loaded coupler.
8. Release the drive shaft spring-loaded coupler by pushing in against the spring-loaded coupler and turn it clockwise.
9. The Hi/Lo motor output shaft spring-loaded coupler must now be released to properly connect with the foot end gear box output shaft.
10. Do this by pushing in against the Hi/Lo motor output shaft spring-loaded coupler and turn it clockwise.
11. The coupling will release and engage the foot end output shaft.
12. Ensure that all components are securely installed.

ASSEMBLY

NOTE: Since there is no Hi/Lo motor for semi-electric Single Crank, Two Crank and manual beds with Hi/Lo functions, the drive shaft will mount directly into the foot end gearbox output shaft.

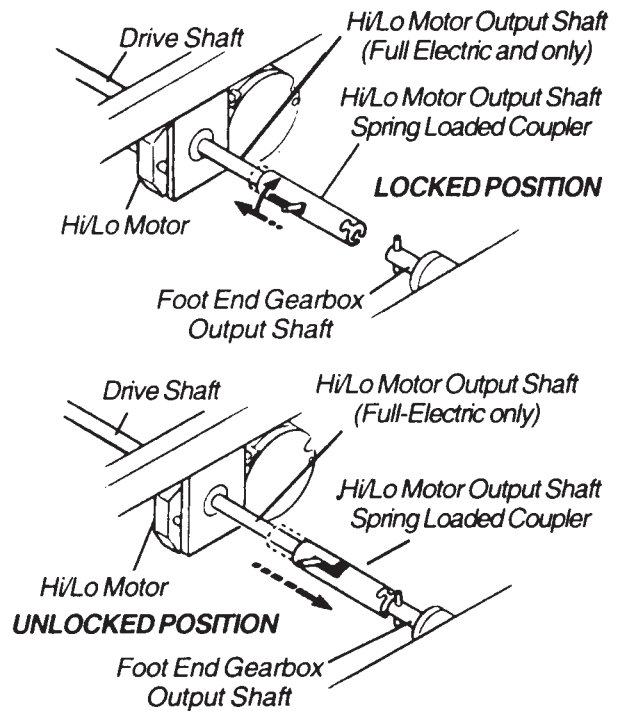
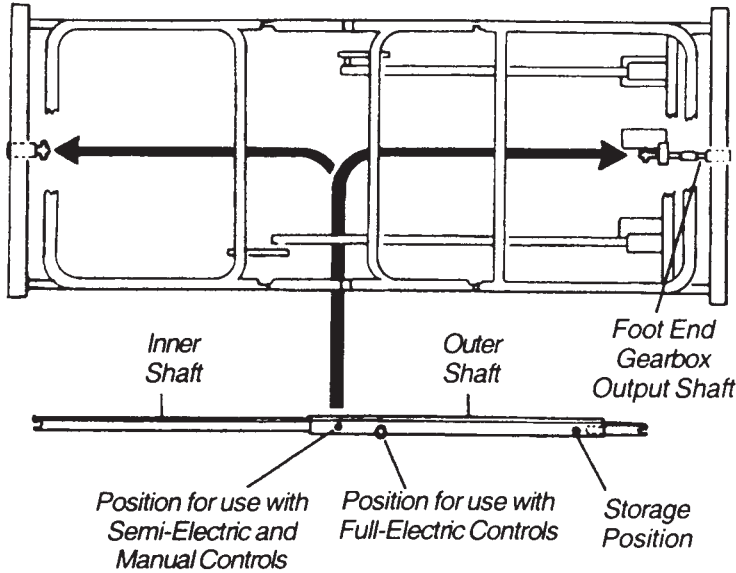
FIGURE 6 - DRIVE SHAFT ASSEMBLY AND INSTALLATION

ACTIVATING THE BED FUNCTIONS (FULL/SEMI ELECTRIC BEDS ONLY) (FIGURE 7)

1. Recheck all electrical connections on junction box at foot end ensuring that all motors are plugged into the correct locations.
2. Plug the junction box cord into a properly grounded 110-volt, 60 cycle outlet.
3. Install pendant holder in convenient place for user.

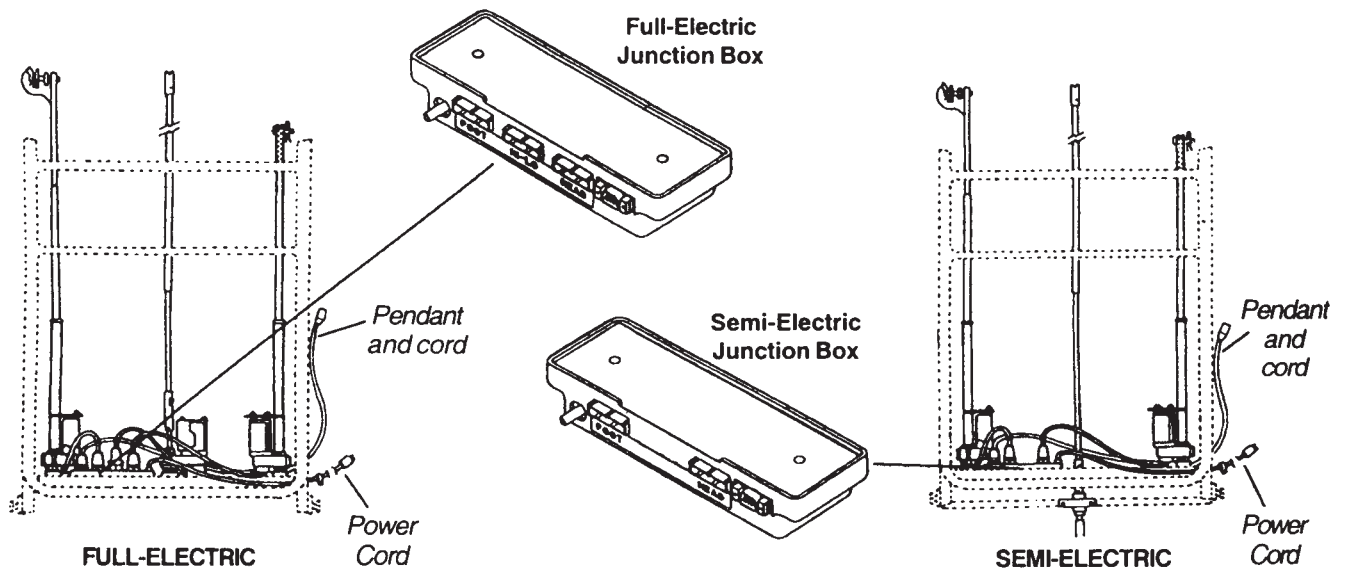
FIGURE 7 - ACTIVATING THE BED FUNCTIONS (FULL/SEMI ELECTRIC BEDS ONLY)

OPERATION AND ADJUSTMENT

This Procedure Includes the following:

Full Electric Operation

Semi-Electric Operation

Manual Bed Operation

FULL-ELECTRIC OPERATION

1. The accompanying pictures detail the full-electric beds and will acquaint the user with the bed functions and controls.
2. Operate the bed through all phases of its operation.
3. If any problems arise during test, recheck all electrical connections and mechanical hook ups.
4. Full-Electric beds use a six (6) function pendant for all bed operation.
5. All Full-Electric beds come with an emergency crank to allow continued operation in the event of a power outage.


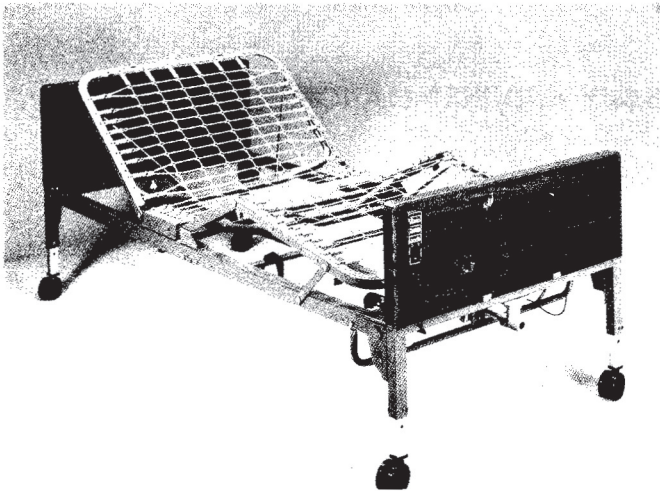
FIGURE 1 - FULL-ELECTRIC BED

Raising and Lowering the Entire Bed (Full-Electric Beds [FIGURE 2])

TO RAISE ENTIRE BED - FULL ELECTRIC BEDS

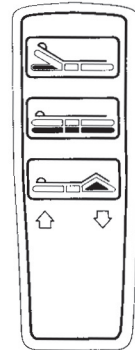Depress button "BED" "UP" as shown on pendant control. This will raise the entire height of the bed.

TO LOWER ENTIRE BED - FULL ELECTRIC BEDS

Depress button "BED" "DOWN" as shown on pendant control. This will lower the entire height of the bed.

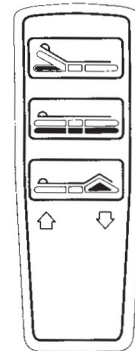
FIGURE 2 - RAISING AND LOWERING THE ENTIRE BED (FULL-ELECTRIC BEDS)

OPERATION AND ADJUSTMENT

Raising and Lowering the Head and Foot Sections (Full-Electric Beds [FIGURE 3])

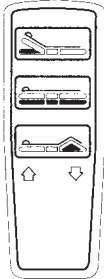
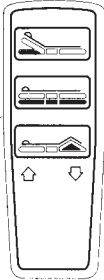
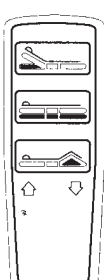
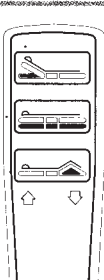
<p>TO RAISE THE HEAD SPRING SECTION FULL ELECTRIC BEDS</p> <p>Depress button "HEAD DOWN" as shown on pendant control. This will lower the head portion of the bed ONLY.</p>	
<p>TO LOWER THE HEAD SPRING SECTION FULL ELECTRIC BEDS</p> <p>Depress button "HEAD DOWN" as shown on pendant control. This will lower the head portion of the bed ONLY.</p>	
<p>TO RAISE THE FOOT SPRING SECTION FULL ELECTRIC BEDS</p> <p>Depress button "FOOT UP" as shown on pendant control. This will raise the foot portion of the bed ONLY.</p>	
<p>TO LOWER THE FOOT SPRING SECTION FULL ELECTRIC BEDS</p> <p>Depress button "FOOT DOWN" as shown on pendant control. This will lower the foot portion of the bed ONLY.</p>	

FIGURE 3 - RAISING AND LOWERING THE HEAD AND FOOT SECTIONS (FULL-ELECTRIC BEDS)

SEMI-ELECTRIC OPERATION

1. The accompanying pictures detail the semi-electric beds with the hi/lo and foot crank locations where applicable.
2. The semi-electric Single and Two Crank beds with Hi/Lo operation use a four (4) function pendant for head and footspring functions and a hand crank for hi/lo bed operation.
3. The semi-electric fixed height beds will feature head and foot section operation but the overall bed height is fixed and cannot be altered.
4. Determine your particular semi-electric bed to assist in the operation of the features of the bed.
5. Operate the bed through all phases of its operation.
6. If any problems arise during test, recheck all electrical connections and mechanical hook ups.
7. All semi-electric beds come with an emergency crank to allow continued operation in the event of a power outage.


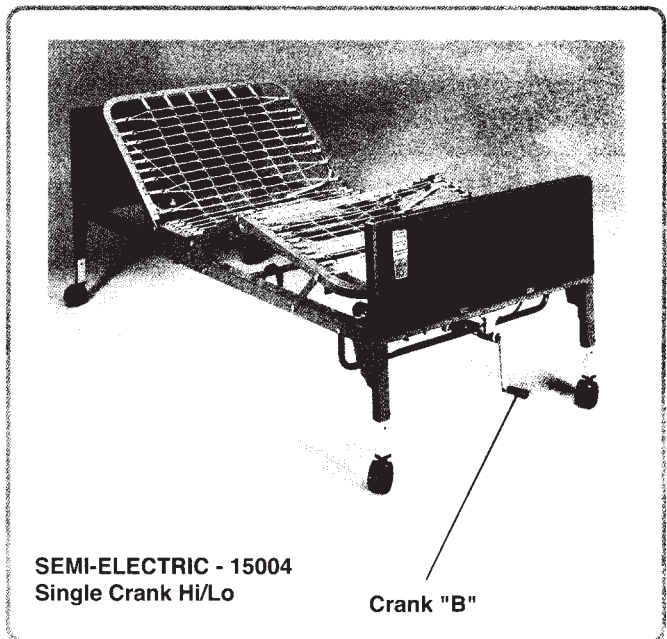
FIGURE 4 - SEMI-ELECTRIC OPERATION

OPERATION AND ADJUSTMENT

Raising and Lowering the Entire Bed (Semi-Electric Beds [FIGURE 5])

TO RAISE ENTIRE BED-SEMI-ELECTRIC BEDS

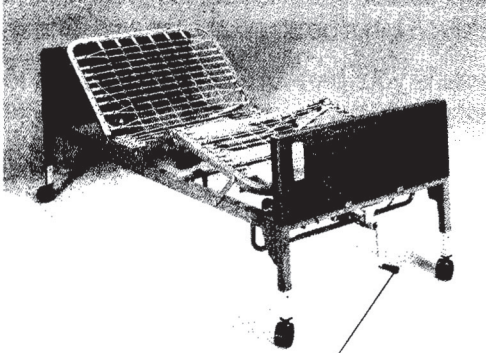
SEMI-ELECTRIC SINGLE CRANK-HI/LO:
Turn crank "B" clock wise. This will raise the entire height of the bed.

Crank "B"

TO LOWER ENTIRE BED - SEMI-ELECTRIC BEDS

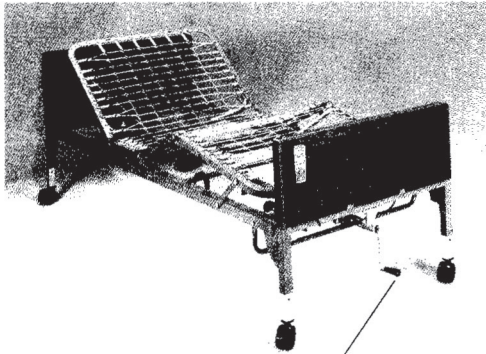SEMI-ELECTRIC SINGLE CRANK-HI/LO:
Turn crank "B" counter clockwise. This will lower the entire height of the bed.

Crank "B"


FIGURE 5 - RAISING AND LOWERING THE ENTIRE BED (SEMI-ELECTRIC BEDS)

Raising and Lowering the Head and Foot Sections (Semi-Electric Beds [FIGURE 6])

TO RAISE HEAD SPRING SECTION - SEMI-ELECTRIC BEDS

SEMI-ELECTRIC SINGLE CRANK -HI/LO:

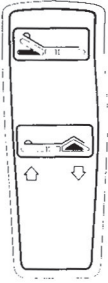
Depress button "HEAD UP" as shown on pendant control. This will raise the head portion of the bed ONLY.

TO LOWER HEAD SPRING SECTION - SEMI-ELECTRIC BEDS

SEMI-ELECTRIC SINGLE CRANK -HI/LO:

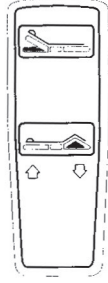Depress button "HEAD DOWN" as shown on pendant control. This will lower the head portion of the bed ONLY.

TO RAISE FOOT SPRING SECTION - SEMI-ELECTRIC BEDS

SEMI-ELECTRIC SINGLE CRANK -HI/LO:

Depress button "FOOT UP" as shown on pendant control. This will raise the foot portion of the bed ONLY.

TO LOWER FOOT SPRING SECTION - SEMI-ELECTRIC BEDS

SEMI-ELECTRIC SINGLE CRANK -HI/LO:

Depress button "FOOT DOWN" as shown on pendant control. This will lower the foot portion of the bed ONLY.

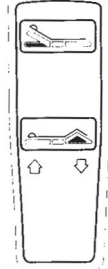
FIGURE 6 - RAISING AND LOWERING THE HEAD AND FOOT SECTIONS(SEMI-ELECTRIC SINGLE AND TWO CRANK HI/LO BEDS)

NOTE: Remember that in the event a full or semi-electric bed should malfunction, it is possible to manually crank the bed using the emergency crank supplied.

OPERATION AND ADJUSTMENT

MANUAL BED OPERATION (FIGURE 7)

1. The accompanying pictures detail the manual bed models and the hi/lo crank locations where applicable.
2. Determine your particular manual bed to assist in the operation of the features of the bed.
3. Operate the bed through all phases of its operation.
4. If any problems arise during test, recheck all mechanical hook ups.


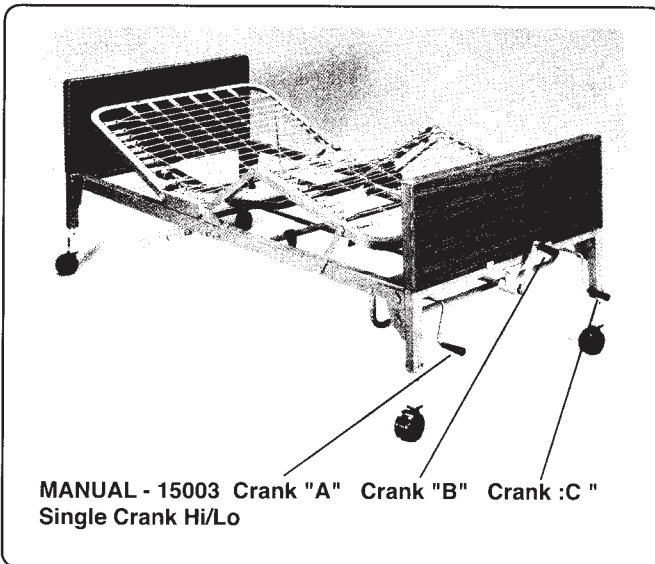
FIGURE 7 - MANUAL BED OPERATION

Raising and Lowering the Entire Bed (Manual Beds [FIGURE 8])

TO RAISE ENTIRE BED - MANUAL BEDS

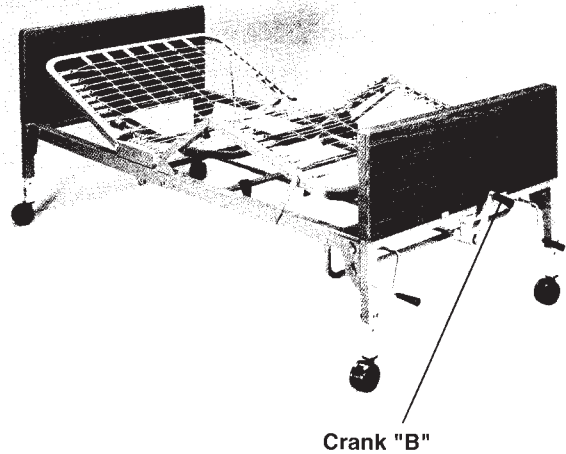MANUAL SINGLE CRANK-HI/LO:
Turn crank "B" clockwise. This will raise the entire height of the bed.

TO LOWER ENTIRE BED - MANUAL BEDS

MANUAL SINGLE CRANK-HI/LO:
Turn crank "B" counterclockwise. This will lower the entire height of the bed.

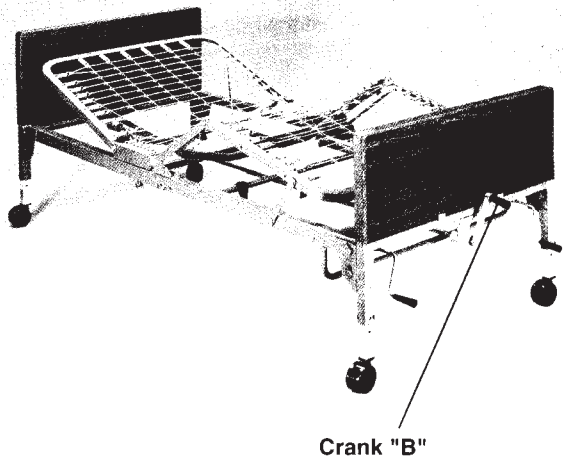
FIGURE 8 - RAISING AND LOWERING THE ENTIRE BED (MANUAL BEDS)

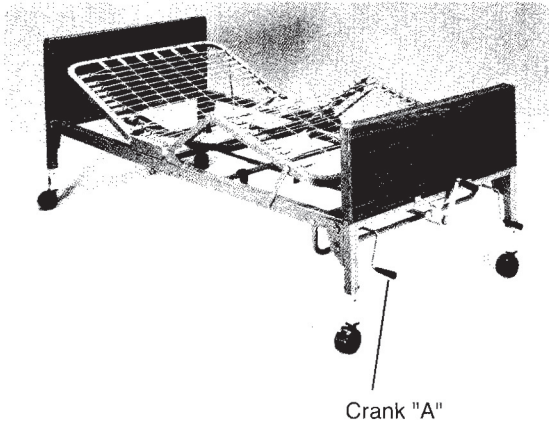
OPERATION AND ADJUSTMENT

Raising and Lowering the Head Section (FIGURE 9 A)

TO RAISE HEAD SPRING SECTION - MANUAL BEDS

MANUAL SINGLE CRANK HI/LO:

Turn crank "A" clockwise. This will raise the head portion of the bed ONLY.

TO LOWER HEAD SPRING SECTION - MANUAL BEDS

MANUAL SINGLE CRANK HI/LO:

Turn crank "A" counterclockwise. This will lower the head portion of the bed ONLY.


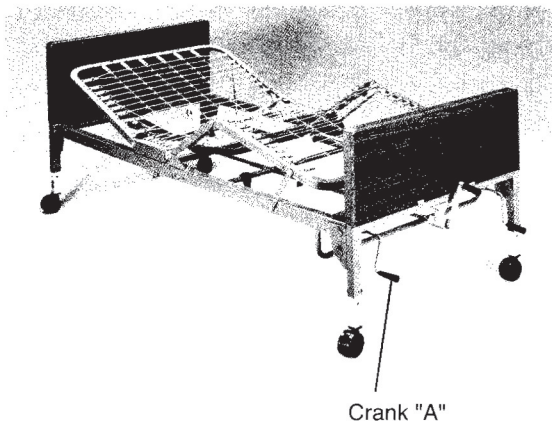
FIGURE 9A - RAISING AND LOWERING THE HEAD SECTION

Raising and Lowering the Foot Section (FIGURE 9 B)

TO RAISE FOOT SPRING SECTION - MANUAL BEDS

MANUAL SINGLE CRANK HI/LO:

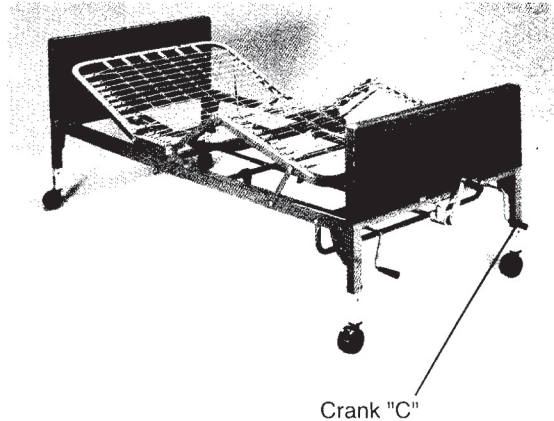Turn crank "C" clockwise. This will raise the foot portion of the bed ONLY.

TO LOWER FOOT SPRING SECTION - MANUAL BEDS

MANUAL SINGLE CRANK HI/LO:

Turn crank "C" counterclockwise. This will lower the foot portion of the bed ONLY.

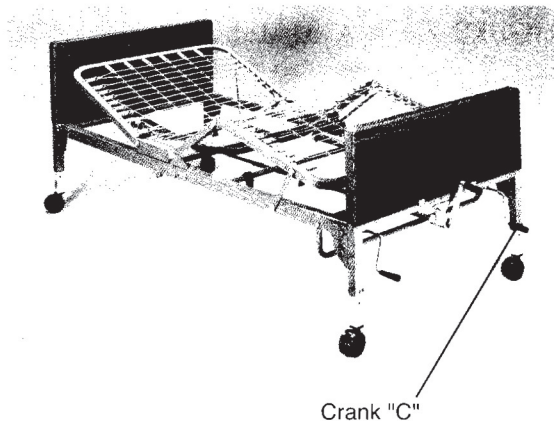
FIGURE 9B - RAISING AND LOWERING THE FOOT SECTION

MAINTENANCE AND SAFETY CHECKS

Drive recommends the following maintenance and cleaning procedures be conducted between users.

ELECTRONICS

- Check all controls to make sure all functions work properly.
 - Foot control
 - Head control
 - Hi/LO
- Check all cables for damaged or frayed wires.
 - Power cord
 - Pendant cord
- Check to make sure all plugs are fully inserted or attached.

Bed frame and sleeping surface

- Visually check all welds
 - Head section
 - Foot section
 - Main Frame
- Check joints between sleeping surface sections for loose fasteners.

Cleaning

- The metal parts of the bed are covered with a powder coating. Clean all coated parts with mild detergent and warm water. Periodically raise head and feet sections of the bed and remove dust from frame. Also, periodically remove mattress and clean mattress deck.

Lubrication and Mechanical

- Lightly grease all actuator screw threads with white lithium grease.
- Lubricate all caster roller and swivel bearings with light machine oil.
- Check all bolts and tighten as needed.

WARRANTY

Lifetime warranty on welds.

Five year warranty on frame.

One year warranty on all other parts and components.

During the warranty period, defective items will be repaired or replaced at Drive's option at no charge.

GENERAL WARNINGS

DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

Read all the instructions before using the bed.

Refer to the owner's manuals for beds and rails for additional product and safety information.

After any adjustments, repair or service and before use, make sure all attaching hardware is tightened securely. After the bed has been assembled, always test to make sure that all sections of the bed are properly and securely in place before using.

ALWAYS use caster locks except when moving the bed.

Body weight should be evenly distributed over the surface of the bed. DO NOT lay, sit or lean in such a way that your entire body weight is placed only on raised head or foot sections of the bed. This includes when assisting the user in repositioning or transferring in or out of bed.

Check all parts for shipping damage and test before using. In case of damage, DO NOT use. Contact a qualified technician for further instruction.

Close supervision is necessary when this bed is used by or near children or people with disabilities.

DO NOT let any individual underneath the bed or in between the raised bed frame components at anytime.

DO NOT permit more than one person on/in the bed at any time.

DO NOT use outdoors.

DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

For Dealers Only - Set-up and Assembly Instructions are in the rear of this manual. These procedures must be performed by a qualified technicians only.

The March 2006 version of the FDA's bed safety guidelines are published by Hospital Bed Safety Workgroup. The latest revision of this document is available at <http://www.fda.gov>.

If the unit is not working properly, call a qualified technician to examine the unit and repair it.
Drive Medical products are specifically designed and manufactured for use in conjunction with Drive Medical

accessories. Accessories designed by other manufacturers have not been tested by Drive Medical and are not recommended for use with Drive Medical products.

Keep all moving parts, including the main frame, mattress deck (head and foot springs/sections) and all drive shafts free of obstruction (i.e. blankets/sheets, heating blankets/pads, tubing, wiring, etc. and other types of products using electric cords which may get tangled around the bed, side rails or legs) during operation of the bed.

Keep the product a minimum of 12 inches away from any direct heat source.

Make sure head and foot springs/sections are connected securely to the universal bed ends before use.

Physically challenged individuals who cannot prevent themselves from rolling/climbing out of the this bed may require alternative safe means of restraint.

Procedures other than those described in this manual must be performed by a qualified technician.

Risk of injury to persons - DO NOT place video equipment such as televisions or computer monitors on bed.

NEVER allow patients to use trapeze or traction units as a total individual weight support.

Trapeze units are to be used only for assisting the patient in repositioning or transferring into or out of bed. Trapeze units must be positioned on a universal bed end as near as possible to the center point of the bed end. The bed is not designed to be used as a patient transport. When transporting a patient, use an approved patient transport. Otherwise, injury or damage may result.

The bed may be equipped with locking casters. When transferring into or out of the bed, always lock the locking caster(s). Inspect for correct locking action on caster locks before using bed. Even with casters properly locked, some flooring surfaces such as tile or wood, will allow the bed to move under some conditions. Use on surfaces such as these must be evaluated by the care provider.

The initial set up of this bed must be performed by a qualified technician.

The total weight limit of the Drive Medical 36-inch (91.4 cm) wide Manual/Electric bed (including accessories, mattress, occupant and any other person/object positioned on the bed) is 450 pounds (204 kg.); 350 pounds (158 kg) patient weight.

Use this bed only for its intended use as described in these instructions. DO NOT use attachments not recommended by the manufacturer.

After raising/lowering the head/foot end of the bed, check the distance between the bottom of the bed rail and the mattress. If there is excessive distance between the bottom of the bed rail and the mattress in which individuals may become entangled, adjust the height of the bed rail (if applicable), or provide alternative means of patient protection.

Once patient assessment concludes that the patient's condition increases the chance of entrapment, the bed **MUST** be in the flat position when left unattended.

Proper patient assessment and monitoring, and proper maintenance and use of equipment is required to reduce the risk of entrapment. Variations in bed rail dimensions, and mattress thickness, size or density could increase the risk of entrapment. Visit the FDA website at <http://www.fda.gov> to learn about the risks of entrapment. Replacement mattresses and bed side rails with dimensions different than the original equipment supplied or specified by the bed frame manufacturer are not interchangeable. Variations in bed side rail design, width and thickness or firmness of the mattress could cause/contribute to entrapment. Use only authorized Drive Medical replacement parts and/or accessories otherwise the warranty is void. Drive Medical will not be responsible for any damage or injury that may result.

To reduce the risk of entrapment make certain that the bed rail cross braces **DO NOT** exceed the width of the mattress. Mattress **MUST** fit bed frame and side rails snugly to reduce the risk of entrapment.

When operating this bed, **ALWAYS** ensure that the individual utilizing the bed is positioned properly within the confines of the bed. **DO NOT** let any extremities protrude over the side or between the bed rails when performing these functions.

Drive Medical recommends that the mattress be centered on the bed frame. Otherwise, individuals may become entangled between the bed rail and the bed frame.

Mattress **MUST** fit bed frame and assist rails snugly to reduce the risk of entrapment.

SAVE THESE INSTRUCTIONS

NOTE: Updated versions of this manual are available on www.Drive Medical.com.

BED RAIL WARNINGS

Assist rails with dimensions different from the original equipment supplied or specified by the bed manufacturer may not be interchangeable and may result in entrapment or other injury.

Although bed rails are not rated to any specific weight limitation, the bed rails may become deformed or broken if excessive side pressure is exerted on the bed rails. The bed rail is not an assist rail for getting into or out of bed. **DO NOT** use the bed rails as push handles when moving the bed.

Always test to make sure that the side rails are properly and securely in place before using the bed.

DO NOT install the optional bed rails without reading and understanding all of the instructions in the instruction sheet that accompanies the bed rail kit.

DO NOT use the side rails as push handles for moving the bed.

These bed rails are intended to prevent an individual from inadvertently rolling out of bed. **DO NOT** use for restraint purposes.

When used with a manual/electrical bed, the bed rails DO NOT fall under any weight limitations. Bed rails can be deformed or broken if excessive side pressure is exerted on the bed rails. These bed rails are used for the purpose of preventing an individual from inadvertently rolling out of bed. The bed rails are NOT intended nor may be used for restraint purposes. If an individual is capable of injuring himself/herself, a physician or a healthcare professional should be consulted for alternative means of safe restraint.

ELECTRICAL WARNINGS

A safety feature of this product includes protection against overheating caused by excessive or extended periods of operation. Depending on the duration, this includes multiple or repeated adjustments or the use of multiple functions at one time. To ensure trouble free operation, always allow a slight pause between multiple adjustments and avoid pressing more than one function button at a time. If thermal protection activation should occur, the bed will not respond to pendant commands. Given this situation, release the pendant button and allow the bed unit to sit for several minutes. This will allow the protection function time to reset and restore bed function. Depending on severity of the initial overheating, this could take up to 30 minutes.

Allow a slight pause between adjustments and avoid pressing multiple buttons at the same time. If pendant buttons are depressed too rapidly or multiple buttons are pressed at the same time, the desired feature may not activate. Simply release the pendant button, permit a slight pause and then activate the next operation.

ALWAYS remove manual crank(s) before performing electronic functions. Otherwise, the crank(s) will turn when the motor is on and could cause personal injury or damage to the bed.

Always unplug the bed from the electrical outlet before cleaning.

DO NOT open assemblies such as the motors, pendant, junction boxes or gear boxes. No user serviceable parts are inside. Only qualified technicians are permitted to repair these parts. If unqualified individuals perform any work on these beds, the warranty is void.

DO NOT place pendant under or between objects. This may unintentionally press the buttons and may cause injury or damage.

Electronic equipment may be influenced by Radio Frequency Interference (RFI). Caution should be exercised with regard to the use of portable communications equipment in the area around such equipment. If RFI causes erratic behavior, unplug the electric bed IMMEDIATELY. Leave unplugged while transmission is in progress. Ensure all cables and cords are routed such that they will not become entangled or pinched. Otherwise damage or injury may result.

If a button on the pendant does not release or sticks, the bed spring will not stop moving.

If a liquid is spilled in or around the electric bed, unplug the electric bed before cleaning. Clean up the spill and allow the electric bed or the area around the electric bed to dry thoroughly before using the electric controls again.

Inspect the covering of the bed's control panel and the patient control panel to assure that the covering is not cracked or damaged.

Keep all electrical cords away from heated or hot surfaces.

Keep the cord away from heated surfaces.

NEVER operate if the unit has a damaged cord or plug. If it is not working properly, call a qualified technician for examination and repair.

NEVER operate this bed if it has a damaged cord or plug, if it is not working properly, if it has been dropped, damaged, or dropped into water. Return the bed to a service center for examination and repair.

On Full Electric beds, the Hi/Lo crank MUST be removed before the bed is used. Failure to remove the crank may cause damage or personal injury.

Refer servicing to qualified personnel only. Grounding reliability depends upon a properly grounded wall outlet. The pendant and power cords must be routed and secured properly to ensure that the cords DO NOT become entangled, pinched and/or severed during operation of the electric bed.

Unplug from the outlet before servicing.

Unplug the power cord from its power source before performing any maintenance on the manual/electric bed. DO NOT unplug the power cord from the junction box. Damage to cord will result.

Use caution when disconnecting the pendant. DO NOT press pendant buttons.

When bed is not to be used for an extended period, unplug electric bed from the wall outlet.

When using an extension cord, use only a three wire extension cord having at least 16 AWG (American Wire Gauge) wire and the same or higher electrical rating as the device being connected. Use of improper extension cord could result in a risk of fire and electric shock. Three to two prong adapters should not be used. Use of three prong adapters can result in improper grounding and present a shock hazard to the user.

When using nasal or masked type oxygen administering equipment, the oxygen or air tubing MUST be routed and secured properly to ensure that the tubing does not become entangled and/or severed during normal operation of the bed.

DO NOT use near explosive gases.

Possible fire hazard when used with oxygen administering equipment other than nasal or masked type.

Use masked or nasal type oxygen administering equipment only in conjunction with this bed. The use of ANY other type of oxygen administering equipment can result in a fire hazard.

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