

Wenzel*ite*Re/habTM
drive

www.drivemedical.com

MSS Tilt & Recline

Item # MS 3000N




EU Authorized Representative
Drive Medical LTD
Ainley's Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JP

www.drivemedical.com

Wenzel*ite*Re/habTM
drive

assembly:

1. Remove all positioning components from box.
2. Remove bolts (A) and insert back mounting bracket (AG) in the following order:
 - Flat retaining bracket (AI)
 - Seat pan (AH)
 - Back mounting bracket (AG)
 - Flat retaining bracket (AI #2)Reinsert bolts.
3. Lift up back to desired angle and seat depth and tighten bolts (A).
4. Insert arm rest assembly into mounting brackets (C).

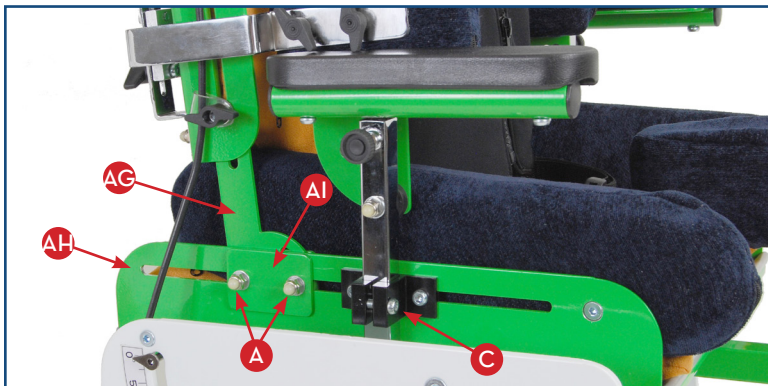

5. Insert laterals in bracket (B) and tighten knob (V)



6. Insert footrest assembly (D) into bracket (F) under the seat of the chair and tighten knob (Y).
7. Mount footrests rods (E) onto vertical section of footrest assembly. Mount left side first.
8. Slide foot sandals onto footrest rods and tighten bracket underneath footrest sandal using knob (X).
9. Insert abductor (G) into round receptacle bracket (Z).




10. Insert headrest post (H) into bracket (I) and turn knob (J) to tighten. Tighten allen screws (K) on triangular bracket (L) and on the back of the headrest cushion (M)



adjustments:

TILT POSITION

Open knob (N) and pull back securing bracket (O). Press tilt actuator (P) and press down of both push handles (Q) to desired tilt angle. Return securing bracket (O) and tighten knob (N).

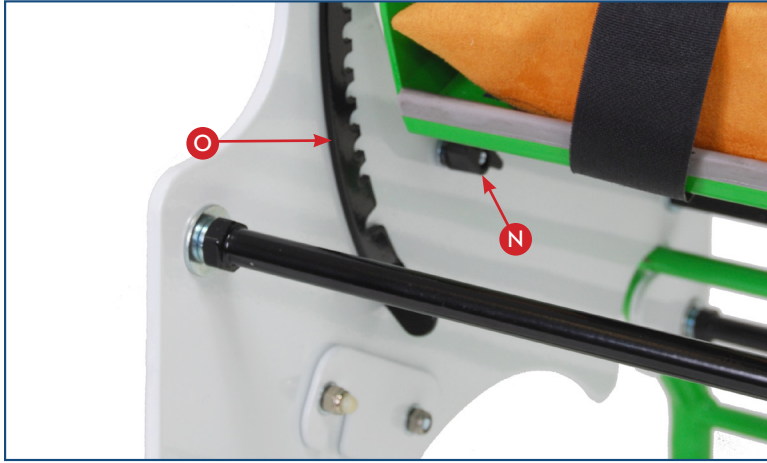

SEAT TO BACK ANGLE AND SEAT DEPTH

Loosen bolts (A) and position back at desired depth. Place seat to back angle at desired setting.




ARM RESTS

Height - Loosen bolt (R) and adjust armrest assembly.

Angle - Push in knob (S) and adjust angle position.

Depth - Loosen bolts (AF) and adjust.

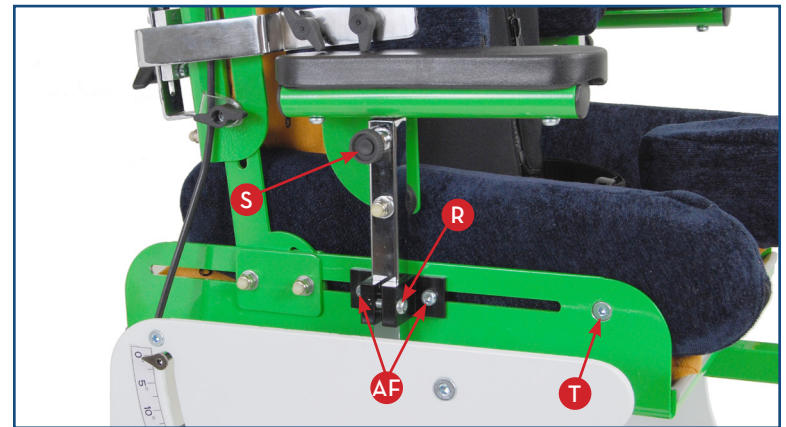

SEAT WIDTH

There are two sets of hip guides included with this product. The set you use will determine the seat width. They can be switched by removing bolt (T) and replacing with the other set. Each set comes with its own bolts.



BACK HEIGHT

Back height can be adjusted by loosening knobs (U) and raising or lowering back.

LATERALS

Laterals can be adjusted in height and width. Loosen knob (V) and adjust height and width.

HEADRESTS

Height


Loosen knob (J) and raise or lower vertical assembly pole.

Depth

Loosen 3 Allen screws (K) on triangular rotating bracket and push/pull headrest to desired position. For additional depth adjustability, mount the vertical assembly pole with the curve either facing inward or outward.

Angle

Loosen 3 Allen screws (M) on circular bracket on headrest and tilt headrest to desired angle. Additional angle adjustability can be achieved by loosening the 3 Allen screws (K) on the triangular rotating bracket and adjust accordingly.

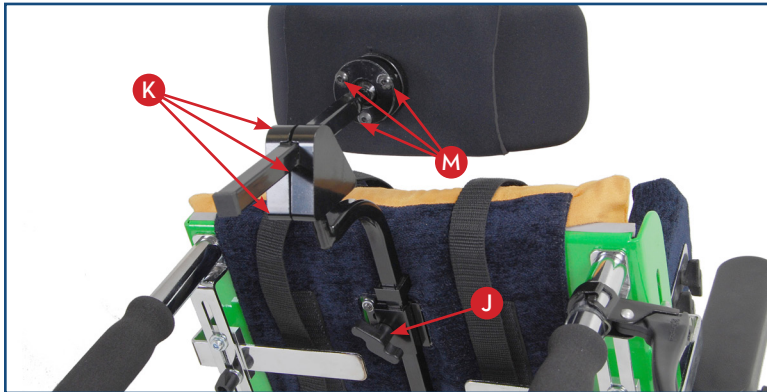


FOOTRESTS

Height - Loosen knob (W) and raise or lower bracket.

Angle - Loosen knob (X).

Position

- Lateral position of foot sandals on rod can be achieved by loosening knob (X).
- Depth position of foot sandals on rod can be achieved by loosening knob (X).
- Depth position of entire footrest assembly can be achieved by loosening knob (Y).

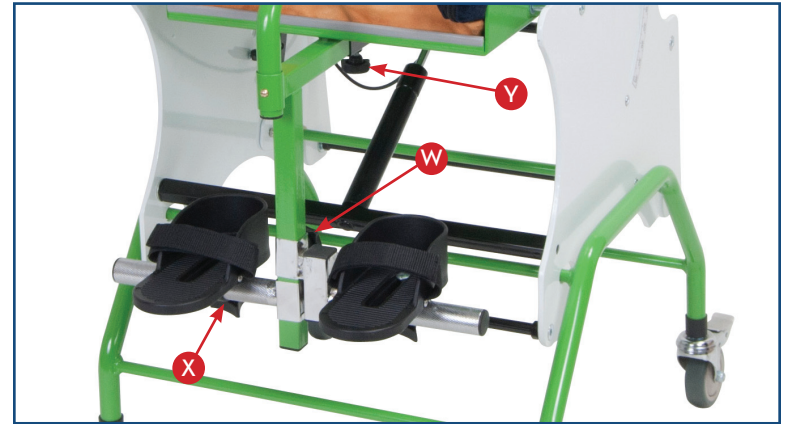

ABDUCTOR

Abductor can be removed, raised or lowered by pressing pushpin (Z). The depth of the abductor can be changed by moving footrest assembly. The abductor pad can be moved by removing bolts under the pad and changing the position of the holes on the pad.



LAP BELT

Length - Use buckle &/or change the position of the Velcro® straps under the seat.

HARNESS

The top and side Velcro® straps on the back of the chair adjust the length of the straps.




ACCESSORIES

BASE

MS 0014N - High base

MS 0015N - Low base

Mount chair onto either base. Lock chair onto base by sliding bracket (AC) over cross bar (AD).

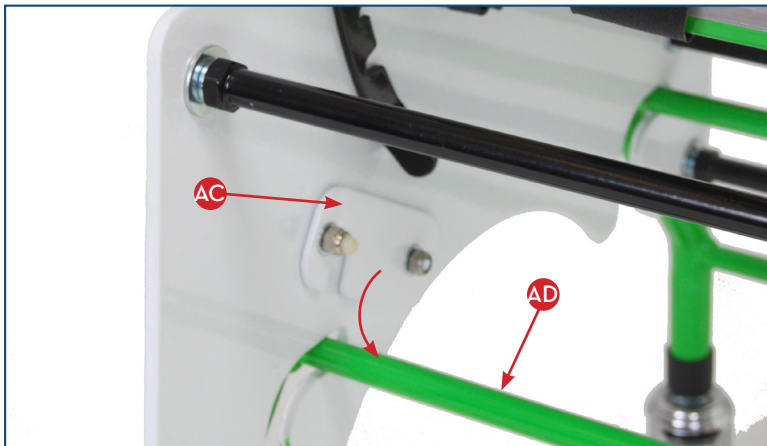

two year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for two years from the date of the original consumer purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This Two Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

*Wenzel*iteRe/hab
drive

© 2012 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in BULGARIA