

## **Elite™ and Omni™**

### **Reclining Bath/Shower/Commode Transfer Systems**

#### **Model Numbers:**

**Elite Bath/Shower Transfer System: 8120, 8130, 8140**

**Omni Bath/Shower/Commode Transfer System: 5120, 5130, 5140**



Elite used as a bath transfer



Omni used as a bath transfer

## ATTENTION

Your Elite and Omni Transfer Systems have been built to the highest standards of quality to ensure years of superior service. Please take the time to read this informative instruction guide and pay particular attention to the safety information provided.

Here are some precautions you should take to reduce the risk of injury:

1. Read and understand ALL instructions.
2. **To reduce the risk of injury or drowning, NEVER leave the user unsupervised or unattended while using this product.**
3. Determine, with a physician or therapist, the suitability of this product for the intended user.
4. Check regularly that all tubing, bolts, buckles, and other parts are tight at ALL times.
5. SAVE THESE INSTRUCTIONS.

Thank you for choosing Columbia Medical.

## Maximum Weight Capacity

To ensure the occupant's safety, DO NOT exceed the following maximum weight limits:

MODEL NO.	MAXIMUM WEIGHT CAPACITY
8120 and 5120	130 lb. (59 kg)
8130 and 5130	175 lb. (79 kg)
8140 and 5140	200 lb. (91 kg)


## BEFORE YOU BEGIN

The Elite or Omni Transfer Systems is shipped partially unassembled. Inside the package, you will find these following pieces:

- Back Frame
- Calf Frame
- Adjustment Pull Handle(s)
- 4 Casters
- Cover and Hardware Kit
- 4 Leg Tubes
- Seating Frame
- Rolling Base
- Transfer Frame



Back and Calf Frames

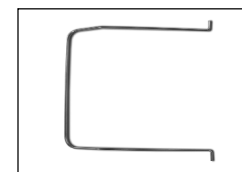

Adjustment Pull Handle




Casters




Cover and Hardware Kit




Leg Tubes

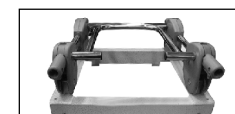

Elite or Omni Seating Shuttle Frame

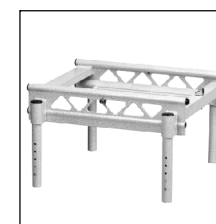


Rolling Base

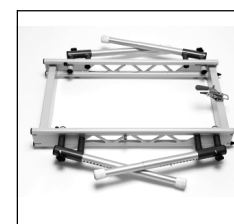

Tub Base

## ASSEMBLY OF SEATING

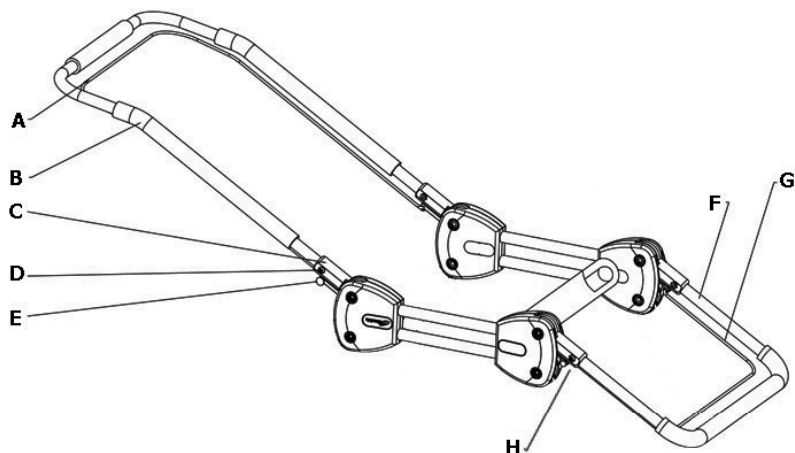

- |                                       |   |
|---------------------------------------|---|
| A - Back angle adjustment pull handle | F - U-shaped calf support frame                             |
| B - U-shaped back support frame       | G - Calf angle adjustment pull handle (straight calf panel) |
| C - Insertion point for back frame    | H - Calf angle adjustment pull tab (bent calf panel)        |
| D - Fastening point for back frame    |   |
| E - Back angle adjustment pull tab    |   |

- One by one, pull on each of the pull-tabs and change the pivot angle to vertical (Fig. 1). Insert the U-shaped stainless steel back and calf frames into the pivots and fasten with bolts and nuts provided.
- Insert the ends of the angle adjustment pull handles into the small holes located on the pull-tab on each side of the frame (Fig. 2).

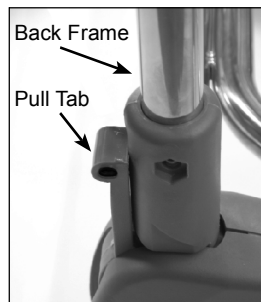


Figure 1

**Firmly press the stainless steel caps over the ends of the pull handle. Please ensure that the end cap fits tightly over the groove on the pull handle. Test it by tugging at the cap. If it comes off, then reinstall.**


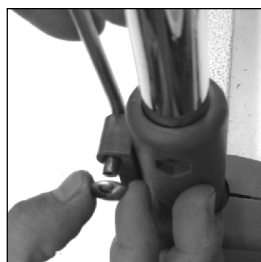


Figure 2

## SAFETY CHECKS

Before using your assembled bath seating, please verify that back and calf supports are properly installed and locked in position. When correctly installed, the pull-tab should be snapped into place and completely engaged into the slot. See the figures below.

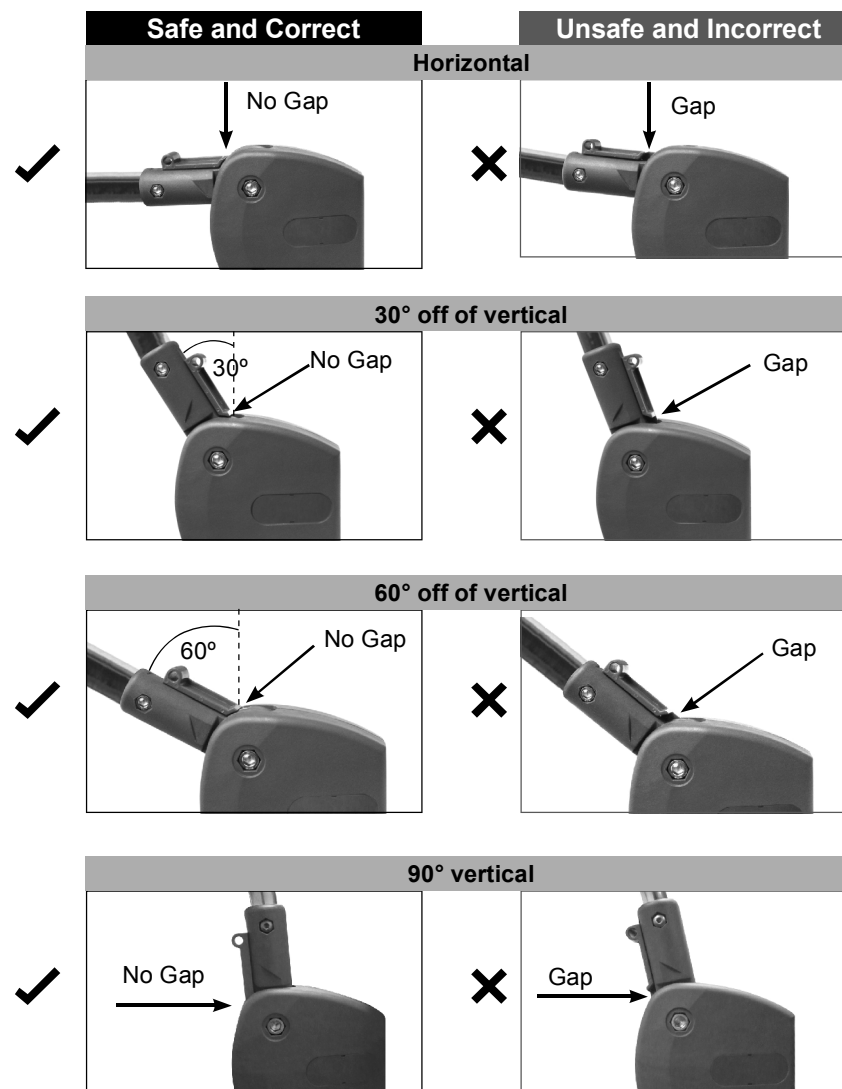

### Cover Installation:

Elite (#8120, 8130, 8140):

- Pull on the angle adjustment pull handles (A, G) to set the backrest and calf supports to the most upright position (Fig. 3). Slip the cover over the backrest and adjustment pull handle and then calf support and pull handle (Fig. 4). Wrap the velcro straps around the top side-tube and secure underneath the seating (Fig. 5).



Figure 3

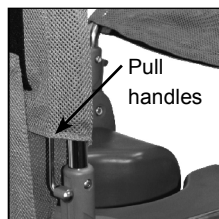


Figure 4

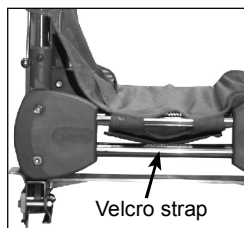


Figure 5

### Adjusting Back Angle:

- Locate the adjustment pull handle (A) on the back of the seating under the cover (it is not necessary to remove the cover to operate the lever). Using both hands, grip the lever and back frame and squeeze together.
- With a lifting motion, adjust the back angle up or down.

**WARNING: Back angle should not be adjusted when the seating is occupied.**

### Adjusting Calf Support Angle:

- On the calf support frame, under the cover, locate the adjustment pull handle (G) for the Elite or the adjustment pull tab (Fig. 1) on each leg for the Omni. It is not necessary to remove the cover. Using both hands, grip the lever and calf support frame and squeeze together for the Elite or pull out the pull tabs for the Omni.
- With a lifting motion, adjust the angle up or down.



Figure 6

### Rolling Base Assembly:

- If ordering footrests, slide the footrest clamp over the leg tubes making sure the notched end faces upward. Align the center slot on the clamp to the holes on rolling base leg (Fig 7). Secure the clamps with bolts. The height is adjustable and can be re-positioned when the seating is fully assembled for correct fit.
- Insert the casters into the socket on the bottom of the individual leg tubes (Fig 8).
- Insert the leg tubes into the rolling base by depressing the locking buttons and sliding the leg tube up the frame.

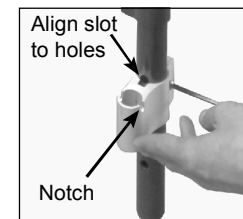


Figure 7

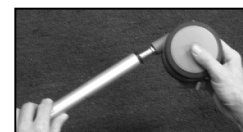


Figure 8

### Seating and Rolling Base (Shuttle) Assembly:

- Insert the guide rail hooks into the guide rail and slide the seating frame assembly onto the rolling base. (Fig. 9). Be sure to slide the seating all the way until the safety latch locks into position on the base (Fig. 10).

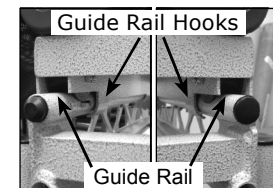


Figure 9



Figure 10

Note: To remove the seating from base, release the safety latch in the back and press the Slider Safe™ button (Fig. 11).

- If ordering with footrests, insert them by aligning the slots on the footrests with the notches onto the footrest clamps.



Figure 11

### Tub Base Assembly:

- Unfold the tub base legs by pulling the legs out and rotating them into position. Each leg is in position when it locks into the slots at the bottom of the frame rail (Fig 12).

Note: When unfolding the legs, grab the leg on the darker-colored pivot section as close to the top as possible. This will prevent damage to the leg and make it easier to unfold.

Note: When folding the tub base for storage, fold the two front legs (the legs closest to the latch) first. The adjusting legs do not have to be removed when folding for storage.

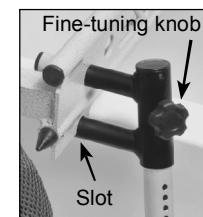


Figure 12

## OPERATING THE TRANSFER SYSTEM

1. With the legs in the open and locked positions, place the transfer base into the shower or tub (Fig 13). Then, roll the seating to the end of the tub base (Fig 14).

Note: The shuttle base is reversible and can be installed on the slide rails in either direction to accommodate any bathroom layout.

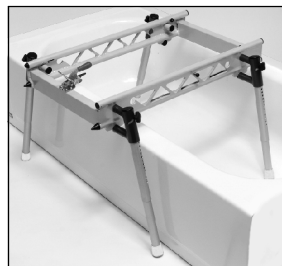


Figure 13

2. Adjust the leg height of the tub base so the frame tubes are at the same height as the rolling base and both are aligned when positioned end to end (Fig. 15).



Figure 14

3. Make rough adjustments ( $\frac{3}{4}$ " ) with the detent buttons (Fig. 15), or make fine adjustments by loosening the back knobs (refer back to Fig. 12) and turning the legs to the desired height. Tighten the adjusting knobs when the height is set.

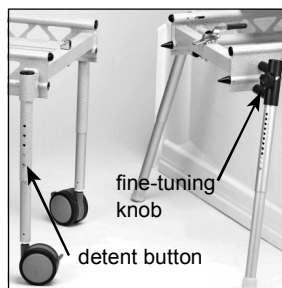


Figure 15

4. When the heights of the tub base and the rolling base are equal, move the rolling base as close as possible to the tub base. Place the tongue of the manual locking latch over the receiver on the tub base (Fig. 16).

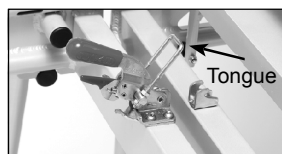


Figure 16

5. Lock the rolling base to the tub base with the center latch before attempting to transfer or slide the rolling base. Loop the tongue from the tub base over the hook receiver on the rolling base. Lock the base to the seating by pulling down on the manual locking handle until the safety lock clicks in place (Fig. 17). The safety lock prevents the seating base from rolling away.

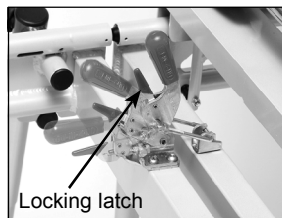


Figure 17

## OPERATING THE TRANSFER SYSTEM (CONT.)

6. Prior to transfer, raise the calf support high enough to clear the sides of the tub (Fig. 18).

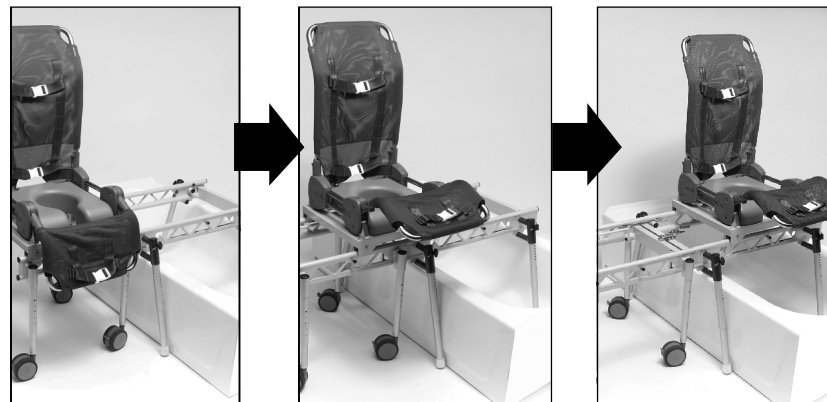


Figure 18

7. When transferring, roll the shuttle base until the latch on the back clicks into the position lock bolt (Fig. 19). There are position lock bolts on rolling base and tub base. These will lock the seating into a secure position and eliminate any side movement and/or rolling during use.



Figure 19

Note: The lock bolt on the tub base can be repositioned into two other locations for better positioning of the individual over the tub in certain applications (Fig. 20).

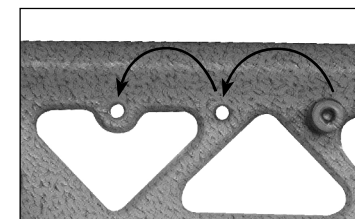


Figure 20

8. To release the lock, pull upward on the black knob on the lower back of the slider and allow it to travel between the coupled rolling base and bath tub base.

## CLEANING INSTRUCTIONS

Soapy water or mild cleansers may be used on all parts of your Elite and Omni Transfer Systems. Harsh abrasive cleansers may dull the finish.


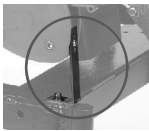








## FABRIC CARE

The fabric cover for Elite and Omni Transfer Systems is made of polyester that is durable, antimicrobial, resilient, quick drying, fade resistant, and machine washable.

Cleaning Instructions:

1. Remove the seating cover by following these steps below:
  - Adjust the back and calf support frames to horizontal. Turn the seating up-side down and remove velcro strap.
  - Slip the cover over the back and calf supports.
2. Using warm water and household laundry detergent, wash the cover in the washing machine using gentle cycle.
3. Air dry.

## ACCESSORIES

MODEL #	DESCRIPTION
5125 	Footrests
5126 	Seat Angle Adjustment Brackets <ul style="list-style-type: none"> <li>• Offer different angle positions</li> <li>• Raises front seat height to 2.5" (63.5 mm), 3" (76.2 mm), or 3.5" (88.9 mm)</li> <li>• For use only with model 8895</li> </ul>
5127 	Rear Anti-Tippers
5150 	Detachable Armrests <ul style="list-style-type: none"> <li>• Offer four height positions</li> </ul>
5151 	Anterior Support Tray (to be used with detachable armrests, model 5150)
8818 	Padded Abductor <ul style="list-style-type: none"> <li>• Attaches easily to bath seating to improve hip alignment and stabilize seating position</li> </ul>
8830 	Lateral Support Pads <ul style="list-style-type: none"> <li>• Designed for better torso positioning</li> </ul>
8835 	Additional Chest and Leg Positioning Belt <ul style="list-style-type: none"> <li>• Quick release strap for various loops on seating, allowing for multiple positions</li> <li>• Two standard positioning belts are included with each seating</li> </ul>
8875 	Head Support <ul style="list-style-type: none"> <li>• Designed for better head positioning</li> </ul>
8834 	Pelvic Positioning Belt



© 2011 Columbia Medical Mfg., LLC

Every Columbia Medical product comes with our one year manufacturer's warranty. This warranty ensures that when you receive our products, they will arrive free of defects in material and workmanship. We will repair or replace any part or entire product at our discretion for a period of up to one year from the date of delivery to the original purchaser. This warranty does not include cost of inconvenience, shipping, property damage, misuse, abuse, accident, or similar incidents. The warranty will not apply if the product has been modified, tampered with, repaired by unauthorized individuals or assembled incorrectly. Custom orders are non-refundable and non-returnable.