

KARMAN R-3600(3-Wheeled Rollator) Limited Warranty/Manual

This warranty is extended only to original purchasers of our products and is not transferable. Karman warrants mainframe of this product against defects in materials and workmanship for one year from purchase date. All components except the main frame are for six months. It is recommended that walker be returned to the authorized dealer through whom it was original purchased.

If within such warranty period, any such product is proven to Karman's satisfaction to be defective; such product shall be repaired or replaced at Karman option. This warranty does not cover shipping costs and labor charge incurred in replacement or repair of this product. You will also be required to provide a receipt of your purchase.


Products which have been subject to negligence, abuse, improper storage or handling, improper operation, unauthorized modifications or misuse are not covered by this warranty. Karman will not warrant the frame or any parts damaged when the maximum weight limit is exceeded. Warranty applies in the U.S. only.

Karman's sole liability extends to repair or replacement of parts or components as previously specified. The foregoing is in lieu of all warranties expressed, implied or statutory, whether written or oral.


OPERATING INSTRUCTIONS

CAUTION: Always request a set up and operating demonstration from your dealer. Never use this walker as an aid when ascending or descending from stairs. Check the condition of this walker periodically.


1. To set up the walker, unfold as much as possible. Then completely unfold by pushing down the metal joint w/ plastic connector separating the two sides of the frame. (See fig. #1)
2. Remove height adjustment screws from handles.
3. You will have two grey plastic pieces with two holes (1x 3/4" and 1x 1/4") which would fit as shown on (fig. #2).
4. Slide handles into frame and insert height adjustable screws firmly to a desired height. Make sure the bolt end fits with the frame's hole (Hexagon).
4. Check both brakes by pushing down on the handles firmly, the brakes should lock. Check if the brakes are stopping to your liking and if not twist bolt in (fig. #3) clockwise.
5. Slide handles into frame and insert height adjustable screws firmly to a desired height. Make sure the bolt end fits with the frame's hole (Hexagon).
6. Check that all wheels are in good running condition (They do not stop suddenly)
7. Check that all screws, nuts, and bolts are tightly fastened.

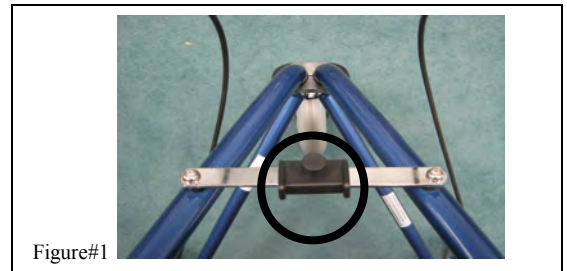

Figure#1

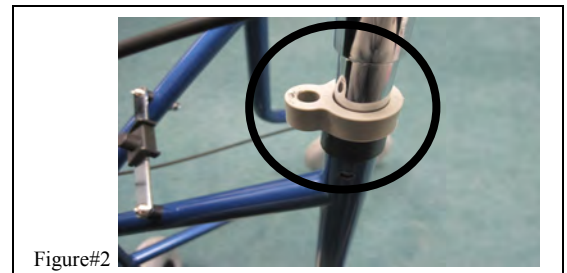

Figure#2

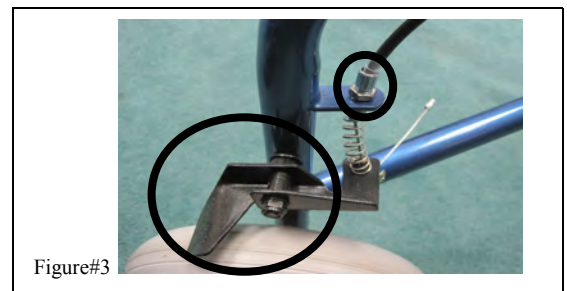

Figure#3



Figure#4

8. To install pouch, secure by attaching the straps over the frame (figure #4). Do this on the left and right hand side of the Rollator. Finally place the grey tray provided in your Rollator box on-top of the basket already installed

**** Brake adjustment for Casters (fig. # 3):**

- 1) While pressing down with one hand on wheel stopper (metal piece), manually twist (Clockwise/Counter clockwise) the nut to tighten or loosen brakes grip on the caster. To ensure that the brake is engaging firmly, apply pressure to the brake lever on the push handle. To prevent brake slippage, please be sure to double check all adjustments.